Ultrasonic & Powder Combination System

Varios Combi Pro

OPERATION
MANUAL

Thank you for purchasing Varios Combi Pro. Please read this Operation Manual carefully before use for operating instructions and care and maintenance guidelines so that you can use it for many years to come through a correct use. Keep this Operation Manual within easy reach of users for future reference.

The reference number of the Operation Manual of the related products is as follows:

| Operation Manual Title | Operation Manual No. |
|---|----------------------|
| Cleaning Adaptor CLA-20 For Prophy Handpiece / For Perio Handpiece | OM-DE0906MA |

– Contents –

| | |
|---|----|
| 1. User and Intended Use | 1 |
| 2. Precautions for handling and operation | 1 |
| 3. Accessory List | 5 |
| 4. Names of Components | 6 |
| 5. Installation | 8 |
| 6. Check before treatment | 11 |
| 7. Using the Ultrasonic Mode | 12 |
| 8. Using the Powder Mode | 16 |
| 9. Post-use Maintenance | 21 |
| 10. Care and Maintenance | 39 |
| 11. Other Functions | 44 |
| 12. Periodical Maintenance Checks | 45 |
| 13. Error Code | 45 |
| 14. Troubleshooting | 46 |
| 15. Specifications | 48 |
| 16. Classification of Equipment | 49 |
| 17. Operation Principle | 49 |
| 18. Symbol | 50 |
| 19. Warranty | 50 |
| 20. Option Parts List | 51 |
| 21. Spare Parts List | 51 |
| 22. Disposing product | 52 |
| 23. EMC Information (Electromagnetic Compatibility Information) | 52 |

* Some sections are indicated with the mark(s) below to show the mode(s) related to the section.

-  **ULTRASONIC** Description regarding Ultrasonic Mode
-  **POWDER PROPHY** Description regarding Prophy Mode under Powder Mode
-  **POWDER PERIO** Description regarding Perio Mode under Powder Mode

1 User and Intended Use

User: Dentist, dental hygienist

Intended Use: Tooth surface cleaning, removal of plaque or calculus, tooth resection, expansion or cleaning of root canal, removal of foreign substances, root canal obturation, enhancing adhesion of fixed restorations, incision or removal of periodontal tissue

2 Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

| Class | Degree of Risk |
|--|---|
|  WARNING | Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed. |
|  CAUTION | Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed. |
| NOTICE | General product specification information highlighted to avoid product malfunction and performance reduction. |

⚠ WARNING

<Ultrasonic Mode and Powder Mode>

- Do not handle the power cord with wet hands. Wet hand contact with electricity may result in an electric shock.
- Keep away from explosive substances and flammable materials.
- If the product overheats or smells of burning, immediately turn off the power and disconnect the main power cord. Contact your Authorized NSK Dealer.
- TO PREVENT ELECTRIC SHOCK, use a main electrical outlet that is earthed.
- Do not operate close to patients with cardiac pacemakers as there is a danger that it may affect the pacemaker.
- The system functions normally in the environment where the temperature is at 0 - 40°C (32 - 104°F), humidity at 35 - 75% RH, and no moisture condensation in the Control Unit. Using in conditions outside these limits may cause malfunction.
- Be careful not to get water or liquid disinfectant in the Control Unit. This could cause short circuits and lead to fire and/or electric shock.
- Repeatedly turning the Main Power Switch ON and OFF may blow a fuse.
- If water droplets, chemicals, solvents or antiseptic solution come into contact with the Control Unit, Handpiece, Handpiece Cord, Handpiece Hose, etc., wipe them off immediately. Failure to do so may result in discoloration or deformation.
- Connect to a public water system that provides drinkable water at a water pressure between 0.1 and 0.3MPa. Using with an incorrect water pressure may result in insufficient water supply, inoperability, or malfunction.
- Use a water softener, etc. to lower the hardness of the water, because using water with a high degree of hardness may cause mineral or limescale deposits in the water circuit, which can lead to problems such as insufficient water levels, operation problems, or breakdown. When doing so, use a Water Tube with a Water Filter to prevent device breakdown due to the foreign material in tap water, and refer to "10-3 Changing the Water Filter" to replace the Water Filter as required.
- Do not sterilize using ultraviolet light. Doing so may result in discoloration of the Handpiece.
- Hold the plug when pulling out the Handpiece Cord or Handpiece Hose. Pulling the Cord or Hose may cause disconnection or damage.

<Ultrasonic Mode>

- Always use the water supply. Insufficient water supply results in overheating, which may damage the surface of the patient's tooth.
- Do not allow the patient or assistant to look directly into the LED of the Handpiece or Handpiece Cord. Doing so may damage eyesight.

<Powder Mode>

- Do not use on the following patients.
 - Those with medical complications or allergies
 - Those who have pre existing conditions (Eg Cardiac, Pulmonary, Renal disturbance or High blood pressure)
 - Those with respiratory diseases.
 - Those with pathologically deep periodontal pockets (6mm or more) or mucosal lesions. Prolonged application of the spray to soft tissue or salivary glands may cause powder air to enter soft tissue or salivary glands, resulting in emphysema.
 - Those with severe digestive ulcers.
 - Those with renal dysfunction.
 - Those with lung or cardiac disturbance or ventilatory impairment.
 - Those with an artificial heart valve or artificial joint
 - Those with allergies. Depending on the patient's constitution, allergy symptoms or stomatitis may be caused in very rare cases. If such symptoms are detected, stop using the product immediately and seek medical attention.
- During use, both the operator and the assistant should always wear protective glasses and protective face masks. Also, continuously vacuum and collect ejected powder during use. If any powder gets into the eyes, eyes wash out immediately with copious amount of water and consult an ophthalmologist.
- Set the supply air pressure to 0.55 - 0.75MPa and use clean, dry air. Using with the incorrect pressure may result in insufficient spray pressure (power), inoperability or malfunction.

<Prophy Mode>

- Do not spray directly onto the cement in the root canal, decalcified enamel, filling, margin of a prosthesis or filling.

⚠ WARNING

<Perio Mode>

- When using Perio Mode, pay attention not to cause emphysema as air containing powder is sprayed into periodontal pockets. If emphysema is caused by treatment using Perio Mode, stop using this product immediately and seek medical attention.
- Do not use on the following patients. Using on periodontal pockets for these patients may result in bacteremia.
 - Patients with endocardial inflammation
 - Patients who have a contagious illness or disease
 - Patients with an immune deficiency disease (neutropenia, granulocytopenia, agranulocytosis, diabetes, hemophilia)
 - Patients undergoing irradiation treatment, chemotherapy, or antibiotic treatment
- Do not use on the following patients. (Excluding the using without the Nozzle Tip)
 - Those with damage or abnormality in the oral cavity.
 - Those with hyperemia, hemorrhage or inflammation in the oral cavity.
 - Those that tend to develop inflammation or sores in the oral mucous membrane.
- Use this product only for maintenance after basic periodontal treatment. Never use it for other purposes.
- Use Perio Mode only on periodontal pockets that meet the following conditions:
 - Periodontal pockets caused by gum disease
 - Periodontal pockets of less than 6mm that have undergone basic periodontal treatment
- Check the depth of periodontal pockets with a measurement probe or photography before use.
- Use the designated cleaning powder only. Use of other powders may result in malfunction, substandard performance or damage to the substantia ossea and dentine, or may cause emphysema. NSK cannot accept any liability for issues that occur as a result of the use of powder other than those designated.
- Nozzle Tip for Perio Mode included in the product package is a sterile item. When using these products, follow the instructions below.
 - Nozzle Tip is a single-use item. Do not reuse nor resterilize it as product breakage or infection could occur.
 - Check before use that the package is NOT open nor damaged. If the package is open or damaged, do not use that Nozzle Tip as it may be damaged or contaminated.
 - Observe the use-by date written on the Nozzle Tip package label. Do not use expired products as product breakage or infection could occur.

⚠ CAUTION

<Ultrasonic Mode and Powder Mode>

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- The end user shall be responsible for any judgment that relates to the application of this product to a patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- This device is for indoor use only.
- Keep the Control Unit on a level surface.
- Do not attempt to disassemble the product nor tamper with the mechanism except as recommended by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product. The Glass Rod and Handpiece may be damaged if the Ultrasonic Handpiece is dropped.
- Operators and all others in the area must wear eye protection and a mask while using this product.
- Should the product function abnormally, cease operation immediately and contact your Authorized NSK Dealer.
- Push the Handpiece securely and fully onto the Handpiece Cord or Handpiece Hose when connecting.
- Do not autoclave the Handpiece Cord or the Handpiece Hose.
- Do not use high acid water or sterilizing solutions to wipe, immerse or clean the product.
- This product is not delivered in a sterile condition, except for the Nozzle Tip (option) and must be autoclaved prior to use. Refer to “9-2 Sterilization” for details.
- Perform regular function and maintenance checks.
- If the product is not used for a long period check it is functioning correctly before using on a patient.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.
- Always replace the fuse with the same specified rating.
- This product is rated Medical Electrical equipment. EMC (Electromagnetic compatibility) is described in the documentation included.
- Installation and use of this product requires special precautions regarding EMC according to the EMC information.
- Portable and mobile RF communications equipment can affect Medical Electrical equipment. Do not use RF equipment near the product.
- The use of ACCESSORIES such as handpieces and cables, other than those specified by the manufacturer, with the exception of handpieces and cables sold by the manufacturer of this product as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of this product.
- This product should not be used adjacent to, or stacked with, other equipment. If adjacent or stacked use is necessary, this product should be observed to verify normal operation in the configuration in which it will be used.

⚠ CAUTION

- The system may present a possibility of malfunction when used in the presence of an electromagnetic interference wave. Do not install the system in the vicinity of any device which emits magnetic waves. Turn off the Main Power Switch of the system as an ultrasonic oscillation device or an electrode knife is located close to the vicinity of use.
- For safety's sake, install the Control Unit in a place where the AC Power Cord can be easily removed. (It is possible to disconnect the control unit from the power source by removing the AC Power Cord.)

<Ultrasonic Mode>

- The temperature of the Tip surface may reach 65°C without using coolant water. To avoid this event, be sure to use coolant water.

<Powder Mode>

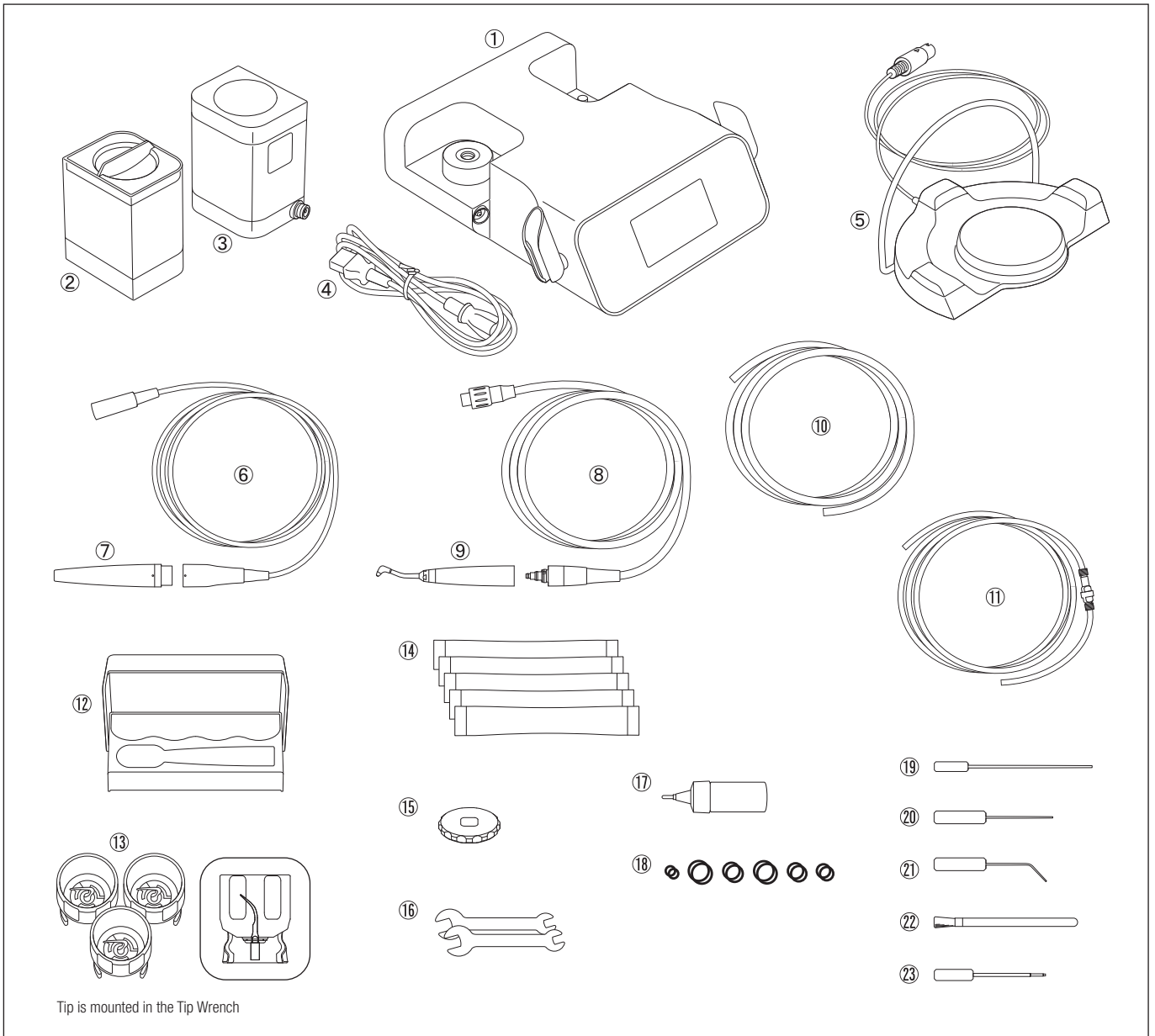
- Be very careful to ensure cleaning powder does not enter the patient's mucosal areas (eyes, nose, etc.) other than the oral cavity. Also, protect the face with a towel or protective glasses, etc., to prevent the cleaning powder from getting into the patient's eyes.
* If the patient wears contact lenses, have the patient wear safety glasses or remove his/her contact lenses before treatment.
- Before connecting the Handpiece Hose, Chamber and Handpiece, or the Chamber and Control Unit, be sure to blow off all water from the connections of the Hose and the Chamber with dry air. Failure to do so may result in moisture entering the air supply, causing solidification of the cleaning powder inside the product.
- Depending on the use environment, the temperature of the Powder Handpiece surface may reach 50°C.
- If too much cleaning powder is put into the Chamber, the proper amount of powder may not be sprayed at the start of use.
- Powder scatters easily because it is very fine. Fill the Chamber slowly and be sure to prevent the powder from scattering.
- Powder stored for a long time in opened packaging may become moist and clog. Always use new, dry powder. When using bottled powder (option), securely fasten the bottle lid after use.
- Apply Vaseline, etc., to the patient's lips to prevent drying or chapping at the corners of the mouth.

NOTICE

- No special training is required for this device.
- Repeated autoclaving may cause the Ultrasonic Handpiece to become discolored due to heat. However, this is due to properties of the product and is not a problem in terms of quality.
- To prevent powder from clogging the pipes, perform the periodical cleaning and maintenance of the vacuum hose, saliva ejector, and separator.

3 Accessory List


| No. | Part Name | Quantity | No. | Part Name | Quantity |
|-----|-----------------------------------|----------|-----|---|----------|
| 1 | Control Unit | 1 | 14 | FLASH pearl | 5 |
| 2 | Irrigation Bottle | 1 | 15 | Ring Wrench | 1 |
| 3 | Chamber (For Prophy use) | 1 | 16 | Spanner Wrench (5×8) | 2 |
| 4 | AC Power Cord | 1 | 17 | Turbine Oil | 1 |
| 5 | Foot Control | 1 | 18 | Spare O-Ring | 6 types |
| 6 | Handpiece Cord | 1 | 19 | Cleaning Wire (L) | 1 |
| 7 | Ultrasonic Handpiece | 1 | 20 | Cleaning Wire (S) | 1 |
| 8 | Handpiece Hose (For Prophy use) | 1 | 21 | Cleaning File | 1 |
| 9 | Powder Handpiece (For Prophy use) | 1 | 22 | Cleaning Brush | 1 |
| 10 | Air Hose | 1 | 23 | Unlocking tool | 1 |
| 11 | Water Filter Set | 1 | 24 | Operation Manual (This manual)* | 1 |
| 12 | Sterilization case | 1 | 25 | Tip-Power Guide* | 1 |
| 13 | Tip and Tip Wrench | 3 | 26 | Tip Card* | 1 |
| | | | 27 | Varios Combi Pro Quick Operation Guide* | 1 |

* Not shown in the illustrations above


NOTICE

- FLASH pearl and Perio-Mate powder (option) used in Powder Mode are referred to as “Powder” in this Operation Manual unless otherwise specified.
- Optional Perio Kit need to be purchased separately in order to use Perio Mode. Refer to “20 Option Parts List” for details of Perio Kit.

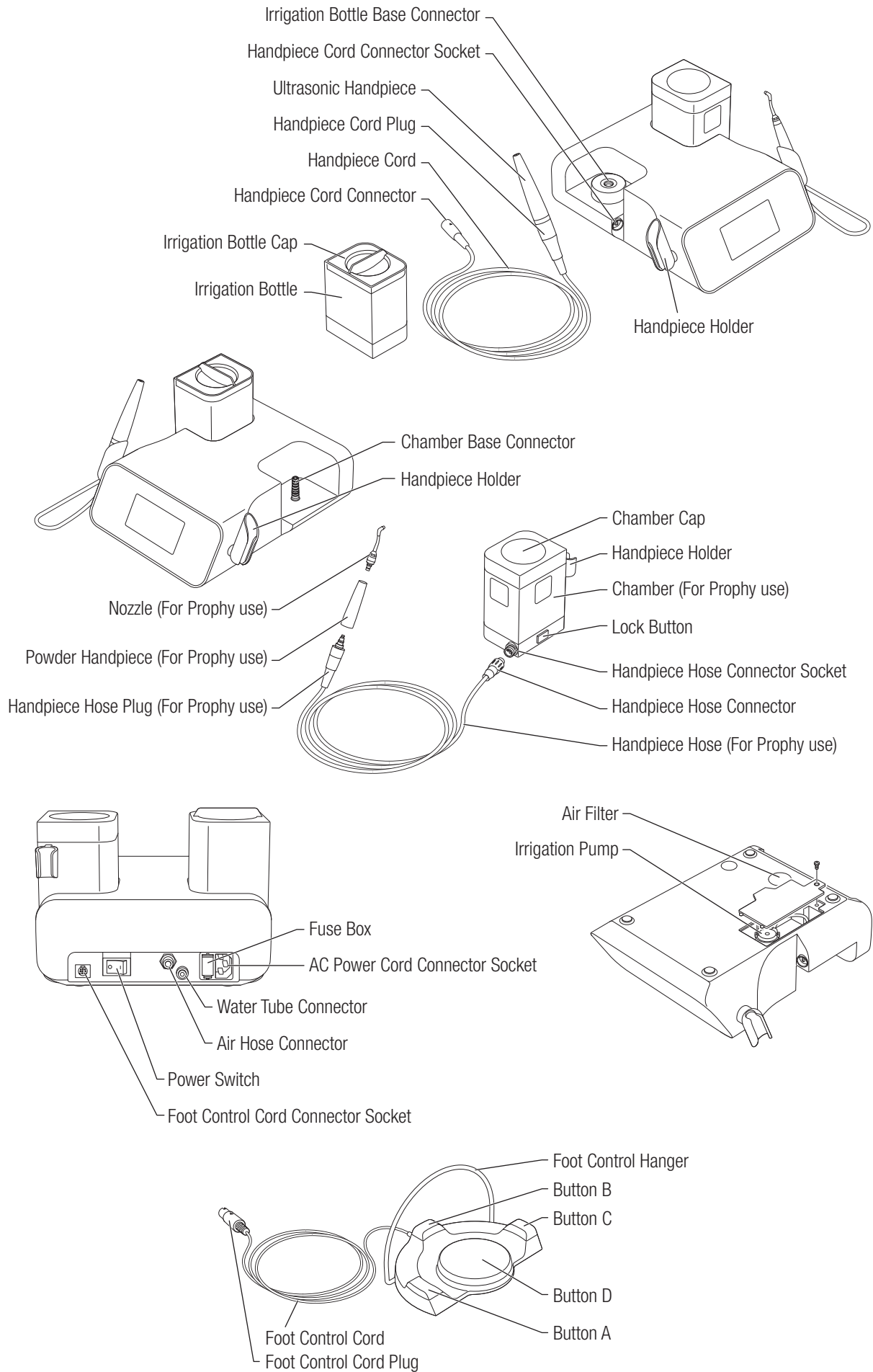4 Names of Components


4-1 Operation Panel

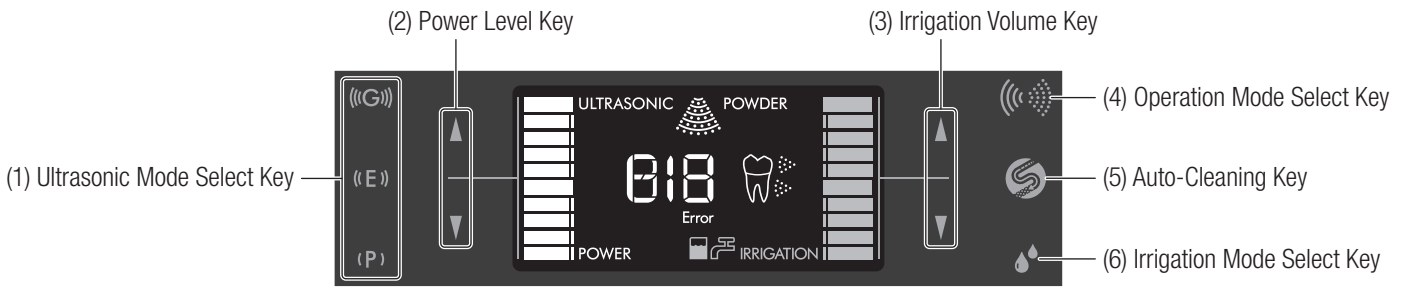
- | | |
|---|---|
| <p>(1) Ultrasonic Mode Select Key Switches the Ultrasonic Mode between P (Perio), E (Endo), G (General). The strongest power level is G, followed by E and P. Power Level, Irrigation Level, and Irrigation Mode can be set for each mode.</p> <p>(2) Power Level Key Sets the power level using the ▲/▼ keys.</p> <p>(3) Irrigation Volume Key Sets the irrigation volume using the ▲/▼ keys.</p> <p>(4) Operation Mode Select Key Switches between “Ultrasonic Mode” and “Powder Mode.”</p> | <p>(5) Auto-Cleaning Key Starts Auto-Cleaning. Refer to “9-1-1-2 Auto-cleaning” (Ultrasonic Handpiece) or “9-1-4-2 Auto-cleaning” (Powder Handpiece) for details. Ultrasonic Mode: Sprays tap water for 15 seconds, then sprays water from the Irrigation Bottle for 30 seconds to clean the Ultrasonic Handpiece. Powder Mode: Sprays air and water at the same time for 10 seconds and then sprays only air for 35 seconds.</p> <p>(6) Irrigation Mode Select Key Switches the Irrigation Mode (Bottle or tap water) for Ultrasonic Mode. Tap water is always used for Powder Mode.</p> |
|---|---|

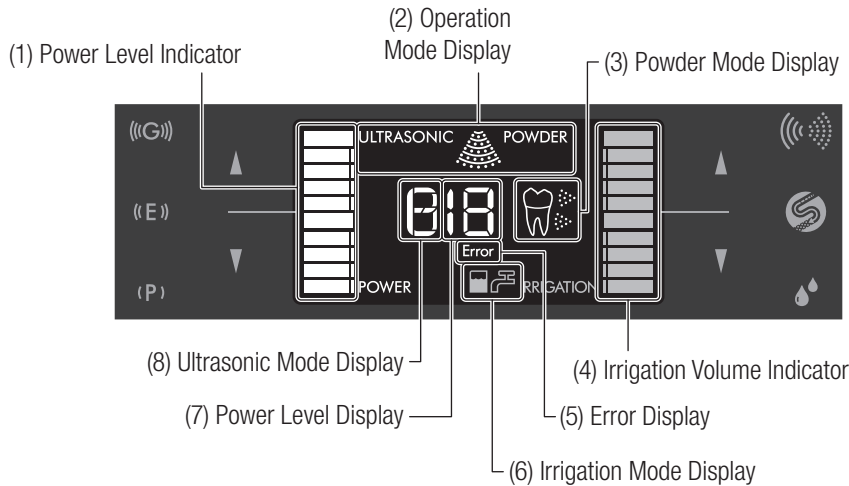4-2 LCD Panel

- | | |
|--|---|
| <p>(1) Power Level Indicator Displays the current power on the Bar Graph Indicator.</p> <p>(2) Operation Mode Display Displays the current Operation Mode (Ultrasonic or Powder).</p> <p>(3) Powder Mode Display Displays the set Chamber (Prophy or Perio) (only when Powder Mode is in use).</p> <p>(4) Irrigation Volume Indicator Displays the selected Irrigation Level on the Bar Graph Indicator.</p> | <p>(5) Error Display Displayed when an error occurs. Refer to “13 Error Code” for details.</p> <p>(6) Irrigation Mode Display Displays the Irrigation Mode (Bottle or tap water) for Ultrasonic Mode. Tap water is always used for Powder Mode.</p> <p>(7) Power Level Display Displays the current Power Level as a number.</p> <p>(8) Ultrasonic Mode Display Displays the current Ultrasonic Mode (P/E/G).</p> |
|--|---|

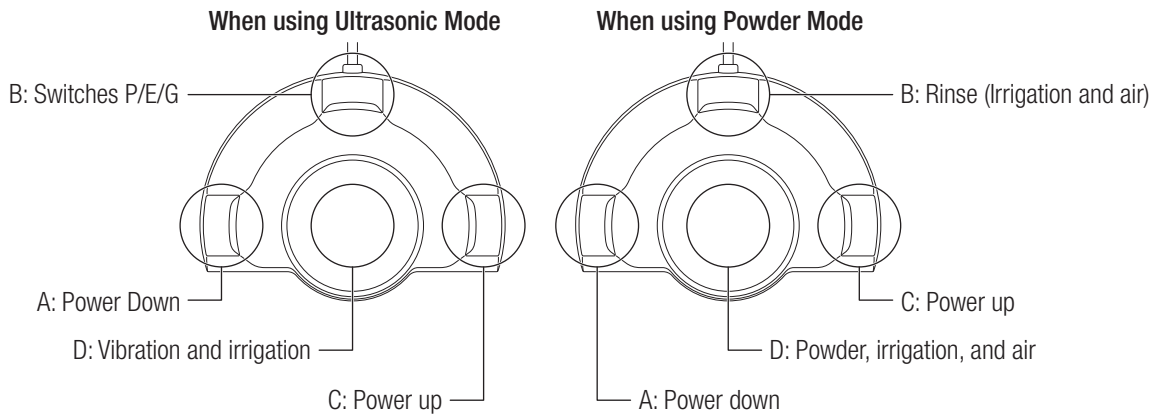4-3 Foot Control

Ultrasonic Mode

- A: Power Down
Reduces the power level.
- B: Switches P/E/G (Ultrasonic Mode)
Toggles Ultrasonic Mode in the order of P (Perio) → E (Endo) → G (General) → P (Perio)... every time the button is depressed.
- C: Power up
Increases the power level.
- D: Vibration and irrigation
Vibrates and irrigates while the button is depressed.

Powder Mode

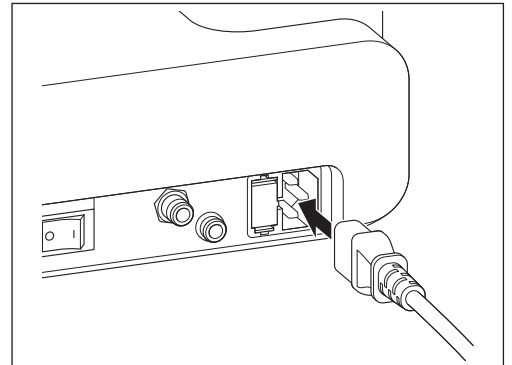
- A: Power Down
Reduces the power level.
- B: Rinse (Irrigation and air)
Sprays water and air while the button is depressed.
- C: Power up
Increases the power level.
- D: Powder, irrigation, and air
Sprays powder, water, and air while the button is depressed.

5 Installation

5-1 Connecting the AC power cord


- 1) Turn the Power Switch to OFF before connecting the AC power cord.
- 2) Insert the AC Power Cord so that it correctly fits into the AC Power Cord Connector Socket at the back of the Control Unit.
- 3) Plug the AC power cord into a wall outlet specified for medical use.

- CAUTION**
- Do not tug the AC power cord by holding and pulling its cord.
 - Make sure that the device has completely stopped before removing or inserting the AC power cord.


5-2 Connecting the Air Hose

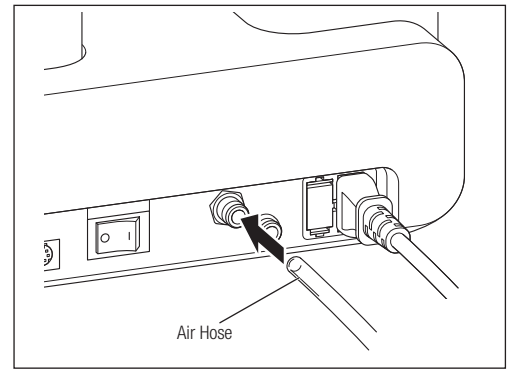1) Insert one end of the Air Hose deep into the Air Hose Connector.

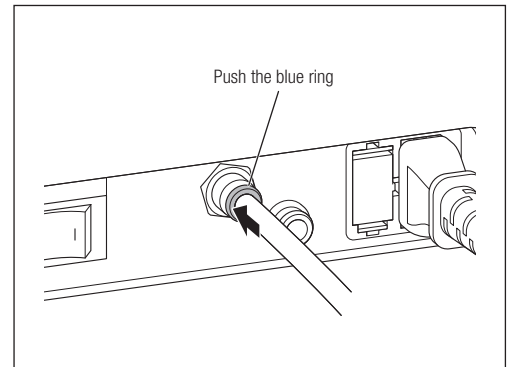
- 2) Pull the Air Hose gently and check that the Hose does not come out.
 3) To remove the Air Hose, pull the Air Hose out while pushing the blue ring toward the Control Unit.


CAUTION • Use clean dry air. Powder may clog if the air contains moisture or oil.

5-3 Connecting the Foot Control

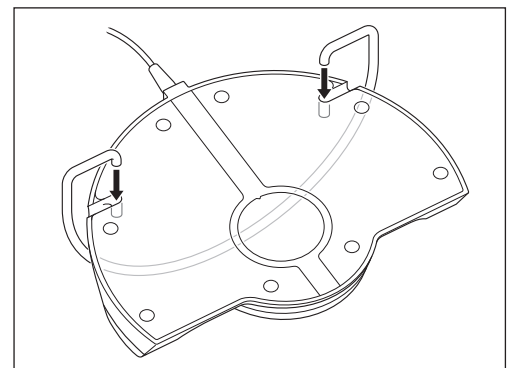1) Insert the Foot Control Hanger so that it fits correctly into the holes at the rear of the Foot Control.

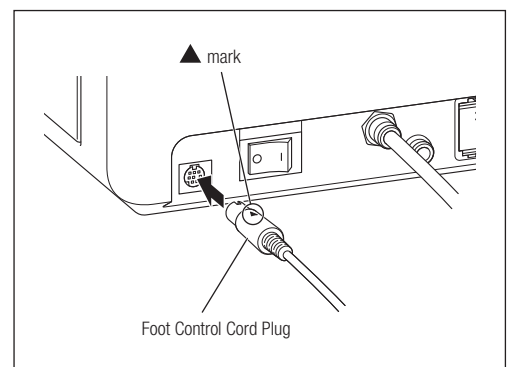
2) Insert the Foot Control Plug into the Foot Control Cord Connector Socket with the ▲ mark facing upwards.

5-4 Connecting the Handpiece Cord (Ultrasonic Handpiece)


Connect the Handpiece Cord Connector to the Handpiece Cord Connector Socket with the ▲ mark facing upwards. Push it in all the way.

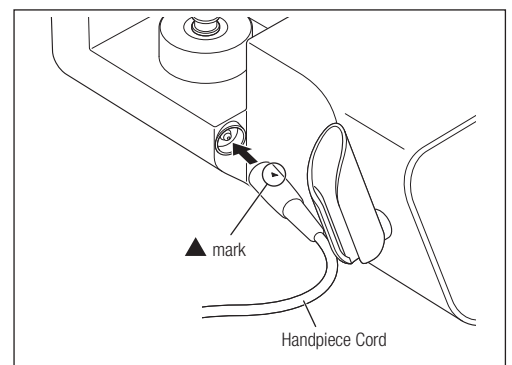CAUTION • Check that the Handpiece Cord Plug is clean and dry before connection.

5-5 Connecting the Handpiece Hose (Powder Handpiece)

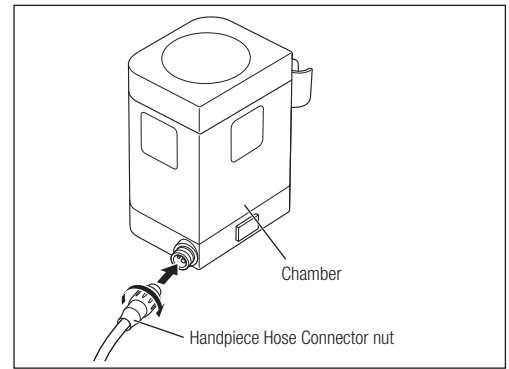- 1) Use air to dry the Handpiece Hose Connector Socket and Handpiece Hose Connector.
- 2) Insert the Handpiece Hose Connector so that it fits correctly into the hole on the Handpiece Hose Connector Socket and fasten the Connector nut.


5-6 Connecting and disconnecting the Handpiece

CAUTION • Be sure to only use the supplied Handpiece.

ULTRASONIC

5-6-1 Ultrasonic Handpiece

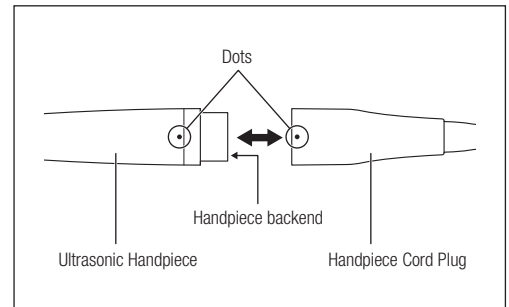- (1) Connection
Align the dots on the Ultrasonic Handpiece and the Handpiece Cord Plug. Push the Handpiece straight into the connector.
- (2) Disconnection
Hold the Ultrasonic Handpiece and the Handpiece Cord Plug and pull the Handpiece out.


WARNING • To avoid an electric shock, do not touch the electrical contacts of the handpiece backend and the handpiece cord.

CAUTION • Do not twist the Handpiece or Handpiece Cord when removing the Handpiece. Doing so may damage the Connector electrical contacts, resulting in malfunction.
• Before connecting the handpiece cord to the handpiece, face the handpiece connector side so light shines into it and check the end of the handpiece. If the light transmitted by the Optic Fiber End Face at the end is uniformly bright, it is functioning normally. If part or all is dark, the Optic Fiber End Face may be cracked or broken.


5-6-2 Powder Handpiece

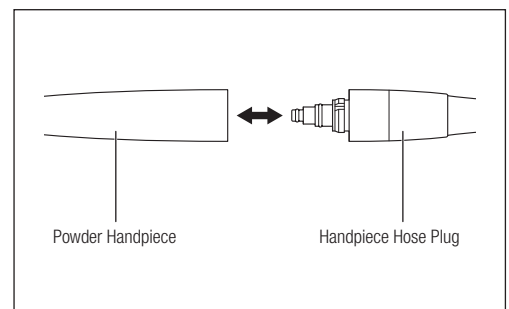- (1) Connection
 - 1) Use air to dry the connection between the Handpiece and Handpiece Hose Plug.
 - 2) Push the Powder Handpiece straight into the Handpiece Hose Plug.
- (2) Disconnection
Hold the Powder Handpiece and the Handpiece Hose Plug, and pull the Handpiece out.


5-7 Connecting the Water Tube

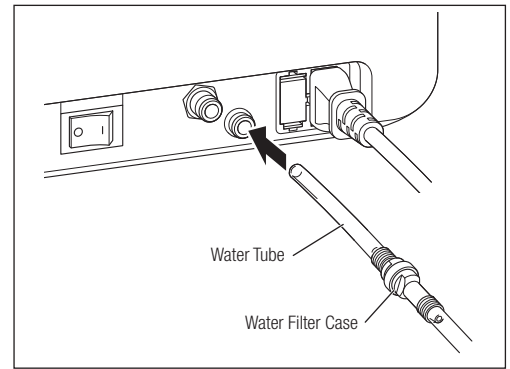- 1) Insert one end of the Water Tube (the end with the water filter) deep into the Water Tube Connector.
- 2) Connect the other end of the Water Tube to the water outlet on the dental unit.
- 3) Pull the Water Tube gently and check that the Tube does not come out.

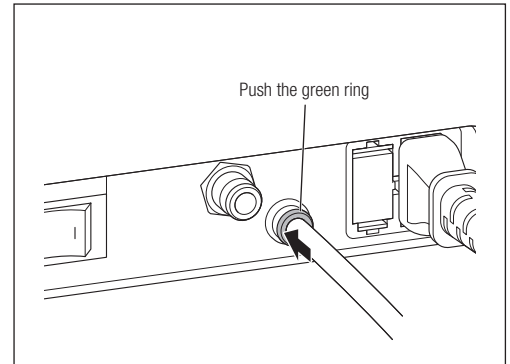- 4) To remove the Water Tube from the Water Tube Connector, pull the Water Tube out while pushing the green ring toward the Control Unit.

WARNING

- Use a water softener, etc. to lower the hardness of the water, because using water with a high degree of hardness may cause mineral or limescale deposits in the water circuit, which can lead to problems such as insufficient water levels, operation problems, or breakdown. When doing so, use a Water Tube with a Water Filter to prevent device breakdown due to the foreign material in tap water, and refer to "10-3 Changing the Water Filter" to replace the Water Filter as required.

CAUTION

- If the water outlet on the dental unit has not been used for a long time, water may contain rust, etc. when it is run for the first time. Make sure that the water is clear before connecting the Water Tube.
- Insert the Water Tube firmly into the Water Tube Connector when connecting it to the Control Unit. Water may leak if the Tube is not inserted correctly.

6 Check before treatment

Before use, operate the product without a patient present to confirm there is no abnormal vibration, noise, heat, etc. Also check the following points.

- Ultrasonic Handpiece: LED lights and water flows.
- Powder Handpiece: Powder, air and water are supplied correctly.

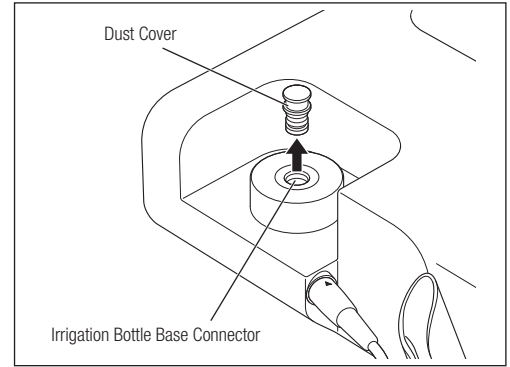
If you notice anything unusual when checking or operating this product, stop using it immediately and contact your authorized NSK dealer. When you perform Auto-Cleaning, the water inside the Handpiece Cord/Hose is ejected, which may make stop water from flowing when using the Control Unit the next time. If that happens, set the Irrigation Volume to 10, and operate the Handpiece for about 30 seconds after water starts to come out. Refer to "14 Troubleshooting" if water does not start to come out after more than 20 seconds, even if you set the Irrigation Volume to 10.

7 Using the Ultrasonic Mode


7-1 Water System Setup

Using the Bottle

- 1) Remove the Dust Cover from the Irrigation Bottle Base Connector.
- 2) Remove the Irrigation Bottle Cap and fill the Bottle water or disinfectant solution to the desired level.

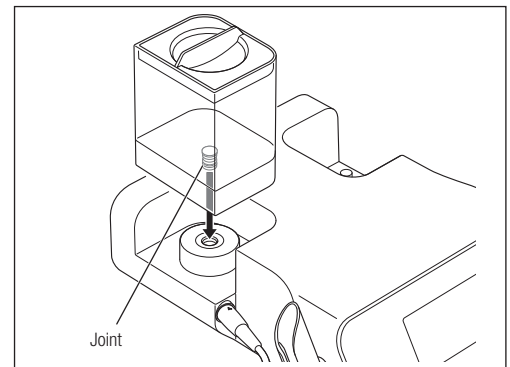- 3) Close the Irrigation Bottle Cap securely and insert the Bottle Joint straight into the Irrigation Bottle Base Connector. Pull the Irrigation Bottle straight up to remove it.

CAUTION

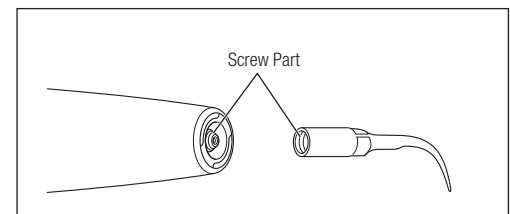
- Only use the dedicated Varios Combi Pro series bottle.
- Insert the Irrigation Bottle straight. Failure to do so may damage the O-ring.
- Clean the Irrigation Bottle and the Irrigation Bottle Cap with clean water after use and before storage.
- Attach the Dust Cover to the Irrigation Bottle Base Connector when the Irrigation Bottle is not mounted.
- Ensure the liquid temperature is below 35°C.
- Do not put strong acids (such as highly acidic water, superacid water, or electrolysis water) or hypochlorous acid water in the Irrigation Bottle as they may cause water clogging or metal corrosion.

Using Tap Water


- 1) Make sure that the Water Tube is firmly connected to the Water Tube Connector of the Control Unit. Refer to “5-7 Connecting the Water Tube” for details.
- 2) Open the water valve of the dental unit and set the water pressure between 0.1 - 0.3MPa (1.0 - 3.0kgf/cm²). If the water pressure exceeds 0.3MPa, use a Water Pressure Reducing Valve (option) to adjust the water pressure to within the specified range. Refer to “20 Option Parts List” for details about the water pressure reducing valve.

7-2 Mounting the Tip


- 1) Check that the screw parts of the Handpiece and Tip are clean.

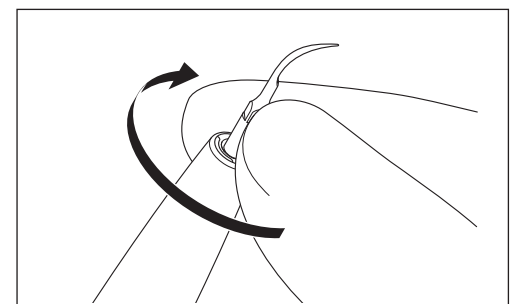- 2) Push the Tip into the distal end of the Handpiece and screw the Tip until it is lightly tightened.

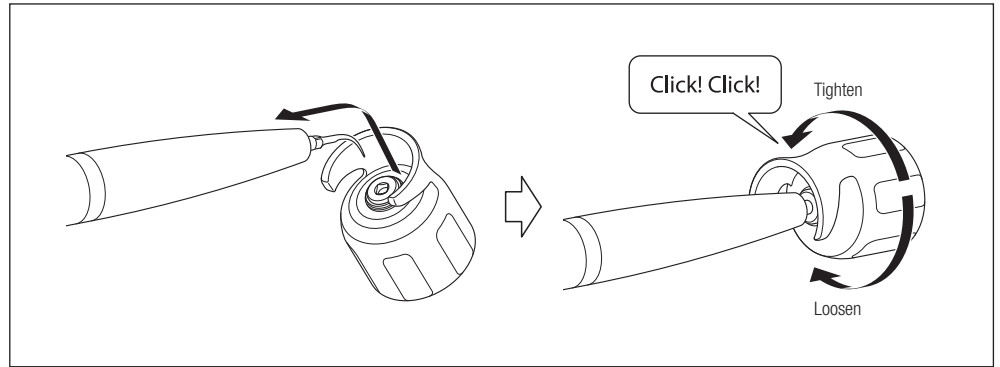
- 3) Match the square shape of the Tip with the hole of the Tip Wrench, then insert the Tip into the hole. Rotate the Wrench until it makes a click sound and cannot rotate any further. During this procedure, hold the Handpiece so that the Handpiece Cord does not become twisted. When removing the Tip, rotate the Wrench in the direction of the "Loosen" arrow.

WARNING

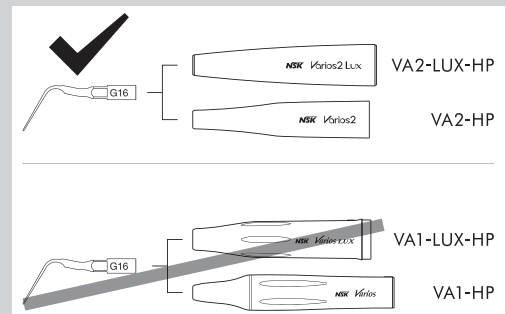- Do not allow contact with gingival, mucosa and/or skin. Doing so may result in injury or burns.
- USE ONLY NSK genuine tips when using this product. Problems including Handpiece damage, failure and accidents resulting from the use of non-NSK tips are not covered in the warranty. The following types of issues may occur when using Non-NSK Tips:
 - Inappropriate vibration caused by incompatible screws.
 - Accidental ingestion of broken tips by the patient.
 - Damage or premature wear of the Handpiece screw
- Do not connect the supplied G16 Tip to a handpiece other than a Varios2 handpiece (VA2-LUX-HP / VA2-HP). Doing so may cause fracturing of the Tip or scratches on the surface of the dentition, prosthesis, etc. due to an increase in vibration amplitude.

CAUTION

- Be careful not to twist the Handpiece Cord when tightening the Tip.
- Be careful when attaching or removing a Tip that is longer than the Tip Wrench, as there is a risk of injury from the point of the Tip.
- Do not use a Tip which has been damaged, bent or corroded as it may break during use.
- To avoid damaging the Tip thread or disengagement of the Tip, always make sure that the Tip is inserted at the correct angle and firmly tightened.
- If debris is found on the Tip thread, clean the Tip immediately. Failure to do so may result in noise or insufficient vibration.
- Do not attempt to sharpen or bend the Tip as doing so may result in the Tip breaking during use, noise or insufficient vibration.
- The Tip is a wear item. Worn Tips have an increased risk of noise, breakage or insufficient vibration. Replace the Tip if noise or insufficient vibration occurs.
- Use the supplied Tip Wrench to mount the Tip. Rotate the Wrench until it makes a click sound and cannot rotate any further. If the Tip is not tightened sufficiently, insufficient vibration or noise may occur.
- When mounting a Tip, always wear sterile surgical gloves and use a sterilized Tip, Handpiece and Tip Wrench.
- Ensure that the power setting does not exceed the recommended Power Level. Failure to do so may result in noise, tip breakage or injury to the patient.
- Do not use Tips that are not designed for removing metal or prosthetic crowns for this purpose. Doing so may result in the tip breaking and falling into the patient's mouth.
- Remove the Tip before attaching or removing the Handpiece or the Handpiece Cord to avoid to avoid injury with the Tip.
- If insufficient vibration is observed during operation, remove the Handpiece from the patient's mouth, and depress the Foot Control again. If this does not improve the condition, the Tip may have loosened during use. Check that the Tip is securely mounted.
- If the Tip becomes loose, makes noise, or does not vibrate sufficiently even though it has been tightened until it makes a click sound, the Tip Wrench may be worn out. The Tip Wrench is a wear item and should be replaced once a year.

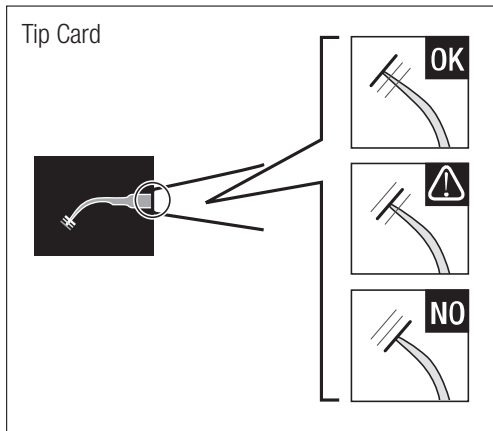
■ Using the Tip Card

- Tips are wear items. Use the supplied Tip Card to check the Tip condition and replace the Tip if necessary.

CAUTION

Place the neck of the Tip in the cut out of the Tip Card. Measure the length of the Tip and check the replacement timing. To guarantee safe and effective use, it is recommended that Tips be replaced with the Tip meets the yellow line (1mm of wear).

Green: No wear - Tip is OK
Tip replacement is not necessary.

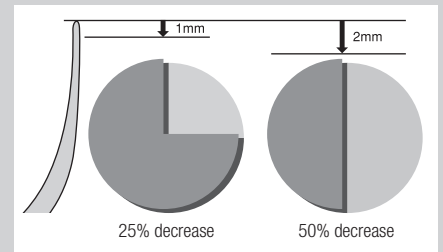
Yellow: Wear of 1mm - Tip is showing some wear
Tip replacement is recommended.

Red: Wear of 2mm - Tip is badly worn
Tip must be replaced.


CAUTION

- Tips are consumables. The efficiency of dental scaling decreases approximately 25% when the top of the Tip wears 1mm and approximately 50% when it wears 2mm. In addition, the vibration condition changes owing to the wear, which may damage a patient's tooth surface. Periodically check the Tip wear condition using the Tip Card, and replace the Tip with a new one in good time.


7-3 Power Level Setting

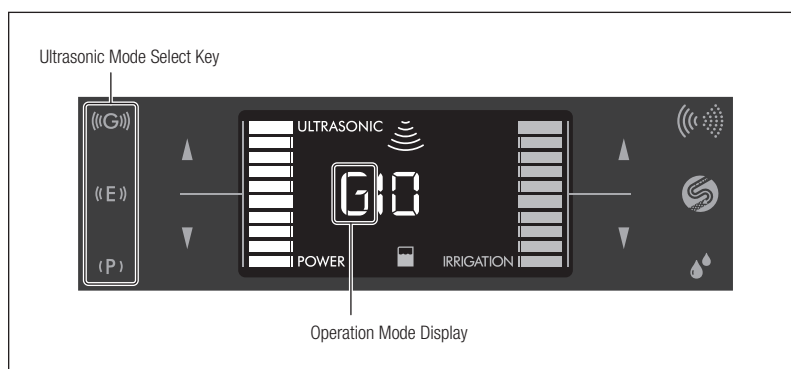- 1) Touch the Ultrasonic Mode Select Key to select the Mode. The selected Mode is displayed on the LCD Panel as P/E/G.

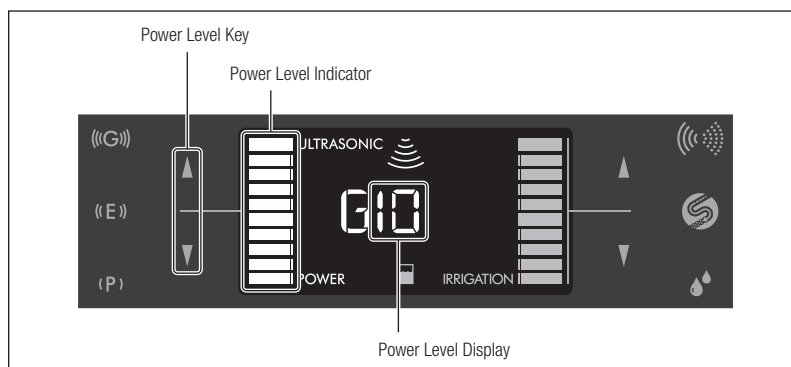- 2) Touch the Power Level Key (▲/▼) to set the Power Level. The Power Level Display and the Power Level Indicator increase or decrease in accordance with the selected level.

CAUTION

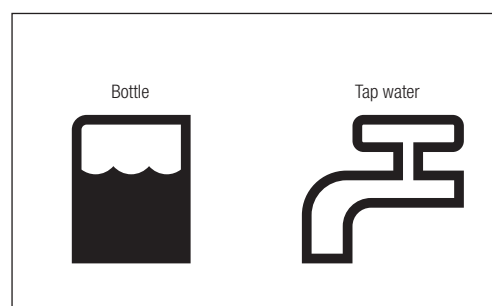
- Ensure the power setting does not exceed the recommended Power Level. (Refer to the supplied "Tip-Power Guide" or the "Tip case label" for the maximum power of each Tip.)

NOTICE

- Touch and hold the Power Level Key to increase or decrease the Power Level continuously. An alarm is heard when the maximum or minimum level is reached.
- To irrigate without vibrating the Tip, set the Power Level to "0 (zero)" and the Irrigation Volume to 1 bar or more.

7-4 Irrigation setting

Touch the Irrigation Mode Select Key to select the Irrigation Mode (Bottle or tap water). The selected Mode is displayed on the LCD Panel as an icon.

7-5 Operation

- 1) Depress the desired button on the Foot Control. A "beep" sound is heard, the Tip starts vibrating and the LED on the Handpiece lights up.
- 2) Adjust the Irrigation Volume by touching the Irrigation Volume Keys (▲/▼) while checking the irrigation from the Tip.

CAUTION

- As a safety feature, this product does not operate when the power switch is turned on while the Foot Control is depressed. In this case, an alarm sound is heard and the Ultrasonic Mode Display (P/E/G) and "ULTRASONIC" blinks alternately. Release the Foot Control and depress it again.
- Always use irrigation. If there is insufficient irrigation the Handpiece will overheat and may injure the surface of the patient's tooth.
- Check that clean water is spraying adequately before use.
- Water may not spray smoothly if the Irrigation Volume is set to a low level. In this case, temporarily increase the Irrigation Volume temporarily and depress the Foot Control. Check that water is spraying smoothly from the Tip and adjust the Irrigation Volume again.
- Do not forcibly rotate the Handpiece Holder as doing so may damage the Holder.

- NOTICE**
- The LED of the Handpiece stays lit for about 5 seconds after the Foot Control is released (Afterglow function).
 - Ultrasonic Mode (P/E/G), Power Level, Irrigation Mode settings and Irrigation Volume are stored after the power is turned off and the same settings are applied when the power is turned on next time (Last Memory function).
 - If the Irrigation Volume is too low with the Irrigation Volume set to 10 or the Irrigation Volume is too high with the Irrigation Volume set to 1 while the Irrigation Mode is set to tap water, adjust the basic Irrigation Level. Refer to “11-3 Basic Water Level Setting” for details.
 - A buzzing sound may be emitted, but this does not indicate a malfunction.

■ During the Handpiece operation :

You can: Adjust the Power Level and Irrigation Volume

You cannot: Change the Ultrasonic Mode or Irrigation Mode settings and perform Auto-Cleaning

7-6 After treatment

Perform maintenance after use on each patient. Refer to “9 Post-use Maintenance” for details.

- This product may malfunction if correct maintenance is not performed.

CAUTION

7-7 Protection Circuit

This product may overheat internally if the control unit is used at power 8 or higher in G mode for an extended amount of time. In this case, the Protection Circuit reduces the Power automatically (Power 7). Bar Graph Indicator flashes from 8 to 10. The indicator stops flashing when the protection circuit is released. However, in this case the Power Level does not increase automatically. Increase the Power Level manually if required.

- NOTICE**
- The Power Level cannot be increased to 8 or higher when the protection circuit has operated (while the bar graph indicator is flashing).
 - The Bar Graph Indicator stops flashing when the Power Level is reduced below 7. It starts flashing again if the Power Level is increased to 7 while the Protection Circuit function is active.

8 Using the Powder Mode

The optional Perio Kit must be purchased in order to use the Perio Mode. Refer to “20 Option Parts List” for details about the Perio Kit.

8-1 Water System Setup

- 1) Make sure that the Water Tube is firmly connected to the Water Tube Connector of the Control Unit. Refer to “5-7 Connecting the Water Tube” for details.
- 2) Open the water valve of the dental unit and set the water pressure between 0.1 - 0.3MPa (1.0 - 3.0kgf/cm²). If the water pressure exceeds 0.3MPa, use a Water Pressure Reducing Valve (option) to adjust the water pressure to within the specified range. Refer to “20 Option Parts List” for details about the water pressure reducing valve.


- Only use drinkable water for irrigation.

CAUTION



8-2 Powder Preparation

- 1) Select the Chamber and powder to be used with this Mode. Fill the Chamber with powder up to the "MAX" line (approximately 80g of FLASH pearl or approximately 50g of Perio-Mate powder).

| Mode | Chamber | Powder |
|-------------|----------------|-------------------|
| Prophy Mode | For Prophy use | FLASH pearl |
| Perio Mode | For Perio use | Perio-Mate powder |

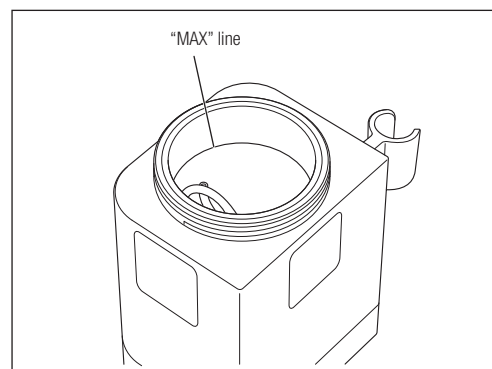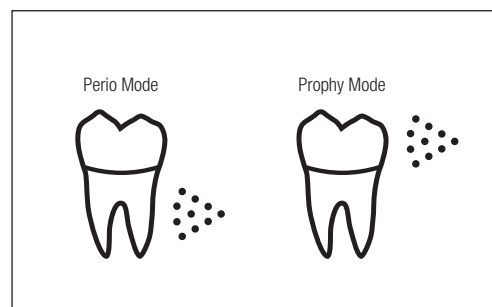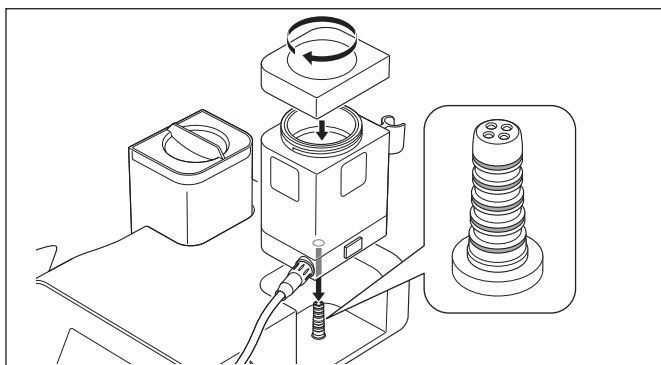- 2) Close the Chamber Cap securely until it fits the Chamber completely and insert the Chamber straight to the Control Unit. The type of inserted Chamber is displayed on the LCD Panel as one of the following icons.

WARNING

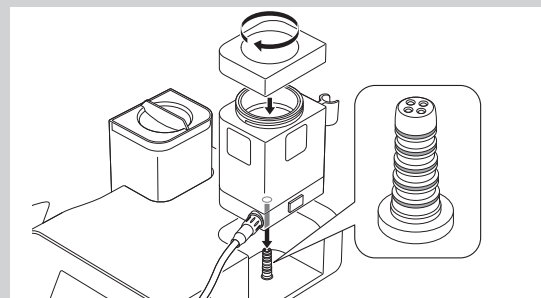
- Use the designated powder only. Using other powders may lead to product failure, damage to the tooth or may cause emphysema. In addition, it may also cause performance reduction of the product. We cannot accept any liability for failures caused by using other powders.

CAUTION

- Securely attach the Chamber Cap to prevent the Chamber Cap from being blown off, which may lead to an injury.

- Before inserting the Chamber into Control Unit, make sure the O-Rings of the Chamber Base Connector are not damaged or torn. Damaged or torn O-Rings could lead to poor operation or malfunction.


- Insert the Chamber into the Control Unit straight. Inserting at an angle could damage the O-Rings, leading to malfunction.
- If too much cleaning powder is put into the Chamber, the correct amount of powder may not be sprayed immediately after starting use.
- Do not use powder that has been stored in an open package or leftover powder. Using such powder may cause clogging of the Nozzle. Do not mix leftover powder with new powder.
- Before inserting the Chamber, check the Chamber Base Connector for water, etc., and wipe it clean if it is wet. Failing to do so may cause powder clogging.
- Do not use discolored powder.
- Always securely close the powder container lid. Failure to do so will attract moisture which may cause clogging.
- If powder adheres to the Chamber and/or the Chamber Cap, clean it immediately or the Cap will become difficult to close.
- Do not invert the Chamber while it contains powder inside. Inverting the Chamber could lead to powder clogging.
- As a safety feature, this product does not operate if the Foot Control is depressed when the Chamber is mounted incorrectly. An alarm sounds heard and the tooth icon and "POWDER" flash alternately. Mount the Chamber correctly.
- The Perio-Mate Powder (option) scatters easily because it is made of very fine particles. Fill the Chamber slowly to prevent the powder from scattering, and be careful not to swallow the powder or allow it to get into eyes.
- FLASH pearl is almost insoluble in water. To avoid clogging of the pipes by using FLASH pearl, periodical cleaning and maintenance of the vacuum hose, saliva ejector and separator are necessary.

8-3 Attaching/removing the Nozzle Tip

8-3-1 Attaching the Nozzle Tip

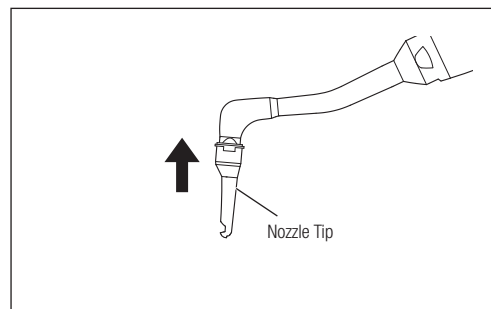Attach the Nozzle Tip (option) securely and firmly to the Nozzle of the Powder Handpiece (for Perio use).


WARNING

- Do not use a deformed or damaged Nozzle Tip. It may detach during use.


CAUTION

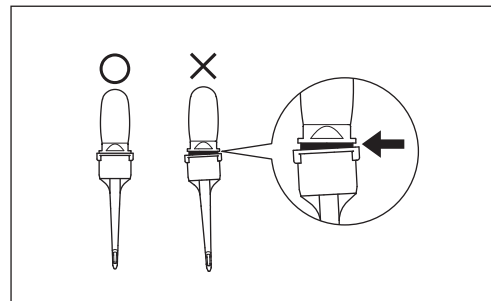- Be sure to attach the Nozzle Tip securely and firmly as illustrated. If the Tip is attached incorrectly, it will fall off and may be accidentally ingested by the patient.
- When attaching a Nozzle Tip to the Powder Handpiece, always wear sterile surgical gloves and make sure that the Powder Handpiece is sterilized.

8-3-2 Removing the Nozzle Tip

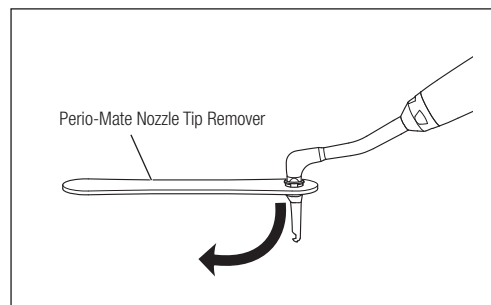Use the Perio-Mate Nozzle Tip Remover (option) to remove the Nozzle Tip and dispose of the Nozzle Tip when removed.


WARNING

- The Nozzle Tip can only be used once. Do not reuse or sterilize the Nozzle Tip as it may be damaged or contaminated. If the Nozzle Tip is reused, it may not operate correctly due to deformation during removal, or it may detach during use resulting in accidental ingestion by the patient. The same applies to a Tip that has been accidentally removed.

8-4 Power Level Setting

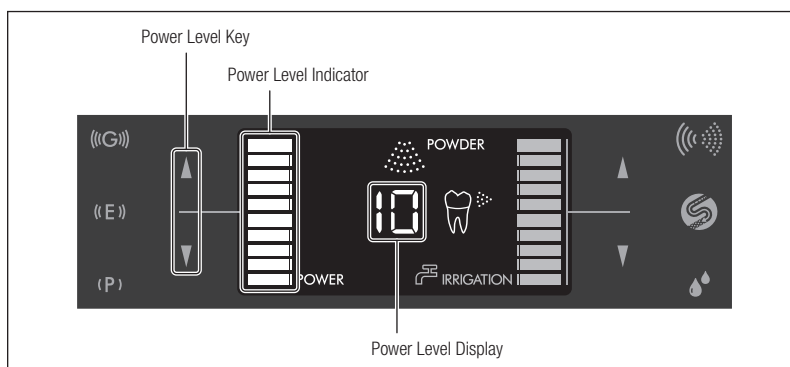Touch the Power Level Keys (▲/▼) to set the Power Level. The Power Level Display and the Power Level Indicator increase or decrease in accordance with the selected Power Level.

NOTICE

- If you specify this setting without attaching the Chamber, the Power Level when using the Prophy mode is set.

8-5 Irrigation setting

- 1) Touch the Irrigation Volume Keys (▲/▼) to adjust the Irrigation Volume.
- 2) Depress the Foot Control to spray powder into a spittoon, etc., to confirm that powder, air and water are appropriately sprayed.
- 3) Adjust the Irrigation Volume while checking the irrigation from the Nozzle.

NOTICE

- If the Irrigation Volume is too low with the Irrigation Volume set to 10 or the Irrigation Volume is too high with the Irrigation Volume set to 1, adjust the basic Irrigation Level. Refer to "11-3 Basic Water Level Setting" for details.
- If you perform adjustment without attaching the Chamber, the Irrigation Volume when using the Prophy mode is set.

8-6 Operation

Powder, water, and air are supplied when the Foot Control button D is depressed. Water and air are supplied when the Foot Control button B is depressed. Observe the operating instructions explained in “8-6-1 Using Prophy Mode” and “8-6-2 Using Perio Mode.”

- WARNING** 
- Never spray powder toward a human body including the operator and the assistant other than to the tooth surface.
 - Stop use immediately if the hose swells or air or powder leaks from connections between the Chamber and Hose during use.
 - Spray water while using this product.
 - Be sure to use a vacuum tube (and a saliva discharge tube together if necessary) to prevent the patient from ingesting a large amount of powder. Avoid continuous use over long periods and allow the patient to rinse out the mouth whenever necessary.

- CAUTION** 
- As a safety feature, this product will not switch on when the Power Switch is turned to ON while the Foot Control is depressed. An alarm sound is heard and the tooth icon and “POWDER” flash alternately. Release the Foot Control then depress it again.
 - Before treatment, adjust the Power Level and the Irrigation Level before using on the patient.
 - Reduce the Power Level when treating around marginal gingiva.
 - Do not spray vertically onto the tooth surface.
 - Maintain the distance between the tip of the Nozzle of the Handpiece to the tooth surface as explained in “8-6-1 Using Prophy Mode” and “8-6-2 Using Perio Mode.”
 - Do not rotate the Handpiece Holder forcibly as it may damage the Holder.

- NOTICE** • A buzzing sound may be emitted, but this does not indicate a malfunction.

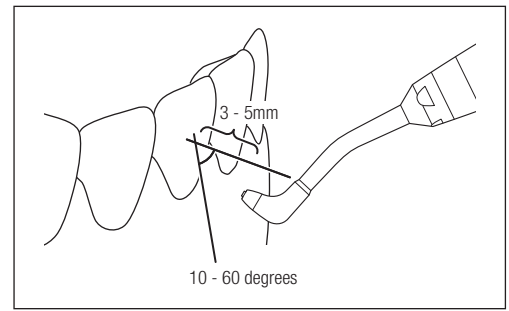
8-6-1 Using Prophy Mode

Keep the Nozzle at a distance of 3 - 5mm from the tooth surface and at an angle of 10 - 60 degrees.

- WARNING** 
- Never point toward soft tissue, gum, or the subgingival area.

- CAUTION** 
- Spray from the root to the incisal edge of the tooth.



8-6-2 Using Perio Mode

■ Using with the Nozzle Tip

Insert the Nozzle Tip deeper than 3mm into the periodontal pocket. Powder may not be sprayed efficiently into the periodontal pocket if the depth is less than 3mm.

■ Using without the Nozzle Tip

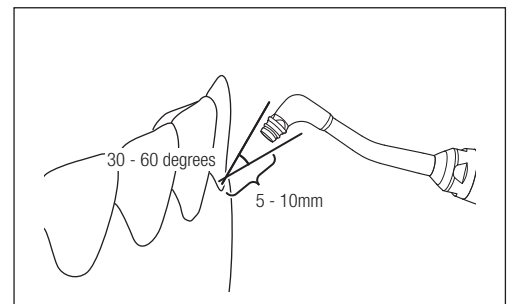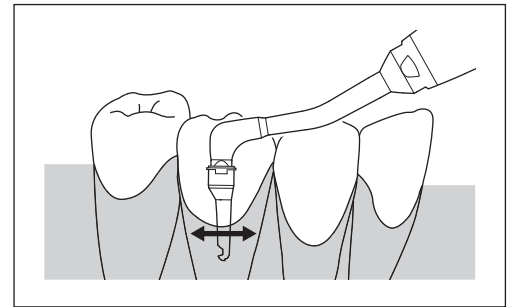Remove the Nozzle Tip from the Nozzle of the Handpiece to remove plaque at the supragingival area and gingival margin, or to remove biofilm and light stains. Keep the Nozzle at a distance of 5 - 10mm from the tooth surface and at an angle of 30 - 60 degrees. Never point the Nozzle directly toward the oral mucous membrane or in periodontal pockets.

**WARNING**

- When using without the Nozzle Tip, never point toward the oral mucous membrane.
- Do not use this product in an area immediately after scaling the tooth surface in a periodontal pocket. Doing so may cause bacteremia.
- Do not insert the Nozzle Tip more than once into the same periodontal pocket during the same treatment.
- Powder may be sprayed for a maximum of 5 seconds in the same periodontal pocket (maximum 20 seconds for one tooth). Never spray powder for longer than the specified time.
- Do not force the Nozzle Tip into a periodontal pocket.
- Do not spray powder while the Nozzle Tip end is located at the bottom of a periodontal pocket. Doing so may cause emphysema.
- Do not use the Nozzle Tip in any places other than periodontal pockets.
- Setting the Power Level too high may cause emphysema. Adjust the Power Level to an adequate level while checking the patient's condition.

**CAUTION**

- The alarm sound is heard every 5 seconds when using Perio Mode.
- In Perio Mode, the Power Level is automatically reduced to a weaker level compared to the same setting for Prophy Mode.

8-6-3 During Handpiece operation

You can: Adjust the Power Level and Water Volume

You cannot: Change Irrigation Mode settings and perform Auto-Cleaning

8-6-4 Releasing residual pressure

The Chamber Cap cannot be opened after operation due to buildup of air pressure inside the Chamber increasing during operation.

Release the pressure remaining in the Chamber before putting powder in the Chamber or removing the Chamber from the Control Unit.

**CAUTION**

- The remaining powder and water comes out of the Handpiece while releasing the residual pressure. Prepare a container such as a cup to catch the powder and water.

- 1) Take the Powder Handpiece out of the Handpiece Holder.
- 2) Place the Handpiece into the container to catch powder and water.
- 3) Touch and hold the Auto-Cleaning Key for about 1 second. Beeps are heard and the residual pressure starts to be released.
- 4) The operation stops when all the residual pressure is released from the Chamber.

8-7 After treatment

Perform maintenance after use on each patient. Refer to "9 Post-use Maintenance" for details.

If the Nozzle Tip (option) is attached to the Nozzle, remove and dispose of it before maintenance.

**CAUTION**

- Failure to perform maintenance may result in malfunction.

NOTICE

- The Power Level and Irrigation Mode settings are stored after the power is turned off (Last Memory function).

9 Post-use Maintenance

Perform the following maintenance after use on each patient.

The following reprocessing instructions are created on the basis of the requirements of ISO 17664.

The effectiveness of the reprocessing through the following procedure has been verified.

Wear eye protection, protective mask, and gloves for safety and infection prevention.

WARNING

- To remove residues, conduct the following maintenance immediately (within 30 minutes) after the treatment. Omitting the maintenance may cause infection, premature failure, and thermal injury resulted from overheating.
- Always obey the local directives, standards, guidelines, etc., as to the cleaning, disinfection, and sterilization of the product.

CAUTION

- Use only the cleaning and disinfection agent products specified in this operation manual.
- Wear eye protection, protective mask, and gloves for safety and infection prevention before starting the maintenance.
- For reprocessing of Tips and the Tip Wrench, refer to the operation manual of the tips (refer to OM-Z0314E).
- For cleaning and disinfection, use disinfectant ethanol or disinfectant isopropanol that does not contain any additives (hereinafter referred to as disinfectant alcohol). Use of disinfectants other than disinfectant alcohol may cause discoloration or cracking.
- Use appropriate disinfectant alcohol according to the regulations and guidelines of each country.
- Do not use solvents such as electrolyzed-oxidizing water (strong acidic water or super acidic water), strong acidic or alkaline chemical, chlorine-containing solvent, benzene and thinner when cleaning, immersing, or wiping.
- After the thermo disinfection, take the product out of the washer-disinfector immediately (within an hour). Failure to do so causes corrosion.
- After the thermo disinfection, dry the product completely. Remaining moisture causes internal corrosion.
- Before the disinfection, be sure to clean the product to remove any residues. If blood remains in the product, it coagulates causing a malfunction of the product.
- Be sure to use the Tip Wrench to mount and remove tips.
- Do not use ultrasonic cleaning to clean Ultrasonic Handpieces.
- Use clean and dry compressed air for cleaning.

9-1 Cleaning and disinfection (each patient)

9-1-1 Ultrasonic Handpiece

CAUTION

- Be sure to perform the auto-cleaning (cleaning of the irrigation tube) after use. If the auto-cleaning is omitted, the irrigation tube may be clogged or metal parts may be rusted.
- Be sure to put distilled water or purified water without impurities into the Irrigation Bottle. The use of water containing impurities causes clogging of the irrigation tube or other problem.
- Make sure that the Ultrasonic Handpiece and Handpiece Cord are securely connected.
- After removing the Irrigation Bottle, dispose of the liquid in it immediately. If liquid remains in the Irrigation Bottle, the liquid may leak from the Irrigation Bottle through the joint section.

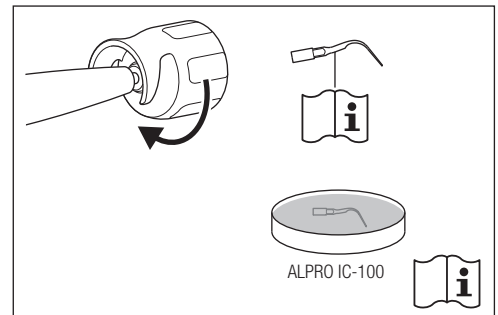
- NOTICE** • The Ultrasonic Handpiece LED does not turn on during auto-cleaning.

9-1-1-1 Removing tip


- 1) Remove the Tip from the Ultrasonic Handpiece.

Remove the Tip from the Tip Wrench.

For reprocessing of Tips and the Tip Wrench, refer to the operation manual that comes with the Tips (refer to OM-Z0314E).

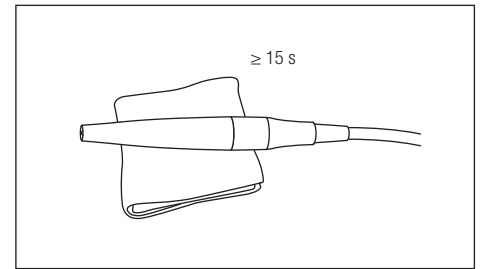- Wipe off dirt from the Ultrasonic Handpiece surface using a lint-free cloth dampened with water.


9-1-1-2 Auto-cleaning


CAUTION

- During auto-cleaning, water used for cleaning comes out of the tip of the Ultrasonic Handpiece. Prepare a container such as a cup to catch the water.

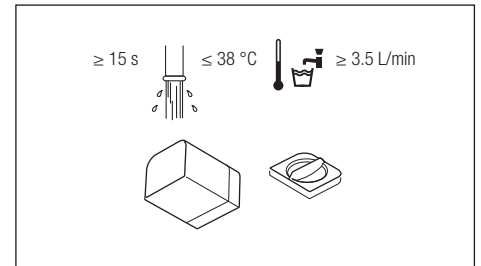
- Select the Irrigation Mode to the tap water irrigation mode and adjust the Irrigation Volume to the maximum.
- Remove the Irrigation Bottle, and then clean the inside of the Irrigation Bottle and the Irrigation Bottle Cap with running water.

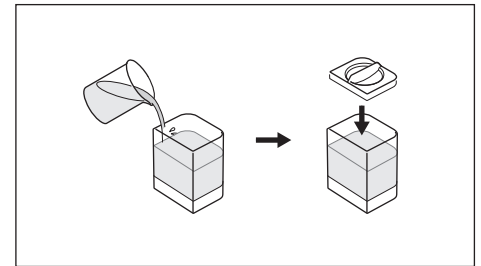- Fill the Irrigation Bottle to the half or more with distilled water or purified water, and then close the Irrigation Bottle Cap securely. (Do not use saline solution.)

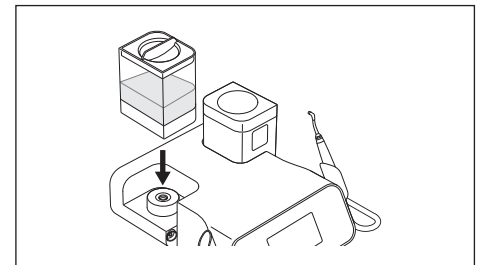
- Insert the Irrigation Bottle straight into the Irrigation Bottle Base Connector.


* If there is a gap between the Irrigation Bottle and the Control Unit, water may leak.

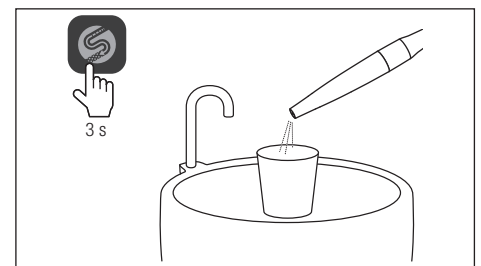- Point the tip of the Ultrasonic Handpiece at a container for catching water, and then touch  for three seconds. Then auto-cleaning starts. (Cleaning time is 45 seconds.)

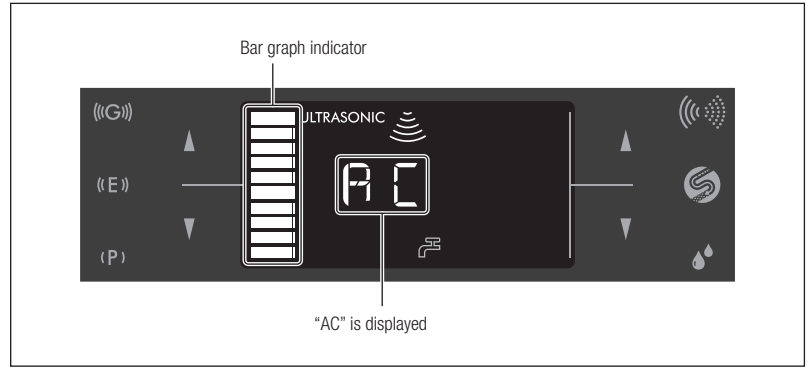
- 6) During the auto-cleaning, "AC" is shown on the LCD.
The bar graph indicator shows the progress.


- * To cancel the auto-cleaning, touch  again.
When the auto-cleaning is complete, the control unit returns to the state before the auto-cleaning.

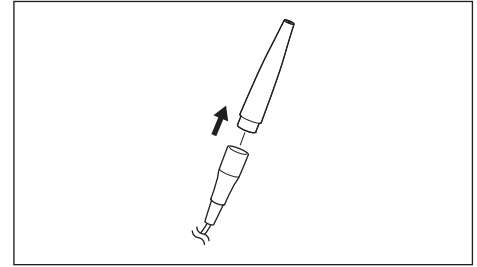- 7) Disconnect the Ultrasonic Handpiece from the Handpiece Cord.


9-1-1-3 Pre-cleaning

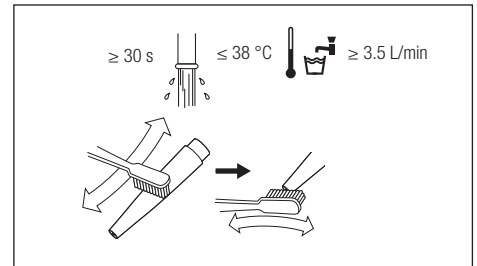
- 1) Clean the Ultrasonic Handpiece using running water and a soft bristle brush.

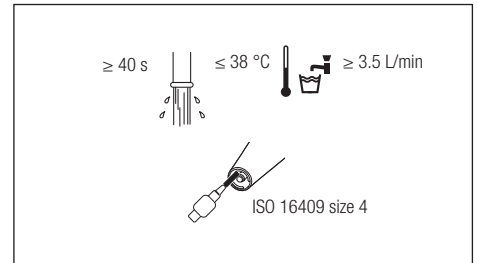
- 2) Clean the tip of the Ultrasonic Handpiece using running water and a brush.

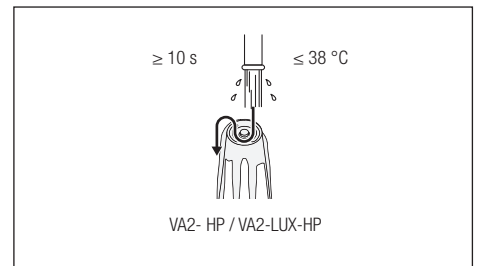
- 3) Rinse the end face of the Ultrasonic Handpiece.

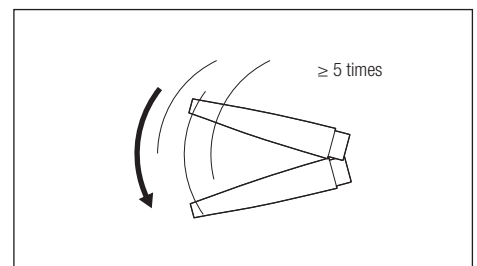- 4) Lightly shake the water off of the Ultrasonic Handpiece.

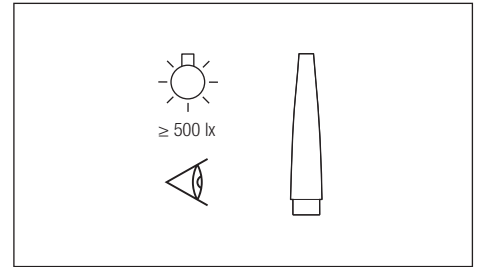5) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 1) until it is removed.

9-1-1-4 Cleaning and disinfection

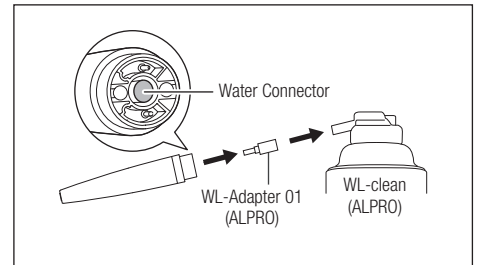
CAUTION

- Be sure to do pre-cleaning before cleaning and disinfection.
- Be sure to carry out disinfection within two hours after cleaning.
- After disinfection, be sure to sterilize the Ultrasonic Handpiece (refer to “9-2 Sterilization”).
- When using any spray (ALPRO), be sure to cover the tip of the Ultrasonic Handpiece with a cloth to prevent detergent and disinfection agent from scattering. We recommend that you use the spray mist absorber (Y900084).
- When spraying the ALPRO spray, keep holding the Ultrasonic handpiece and the WL-Adapter 01.
- When using ALPRO products, refer to the manual of the manufacturer.


Manual

■ Cleaning


1) Attach the WL-Adapter 01 (ALPRO) on to the WL-clean (ALPRO) and connect the Water Connector on the Ultrasonic Handpiece to the end of the WL-Adapter 01 (ALPRO).

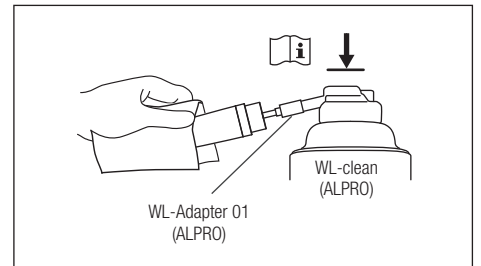2) Spray the WL-clean (ALPRO) while holding the WL-clean (ALPRO) and Ultrasonic Handpiece.

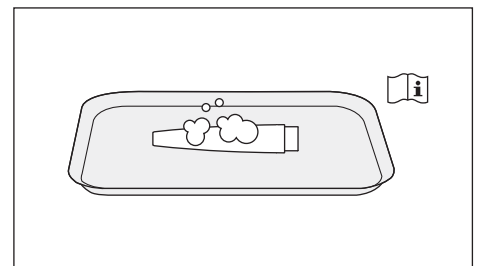
3) Remove the Water Connector on the Ultrasonic Handpiece from the WL-Adapter 01 (ALPRO) and remove the WL-Adapter 01 (ALPRO) from the WL-clean (ALPRO) and then put the Ultrasonic Handpiece on the disinfected tray. Spray the WL-clean (ALPRO) on to the Ultrasonic Handpiece surface, and then wait for the WL-clean (ALPRO) to take effect.


■ Disinfection

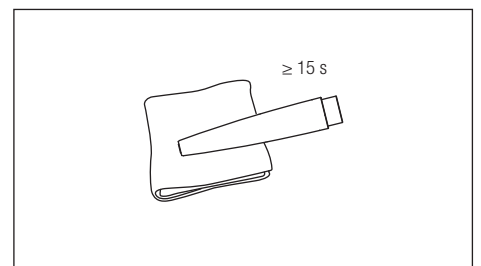
1) Wipe the Ultrasonic Handpiece surface using a lint-free cloth dampened with disinfectant alcohol.

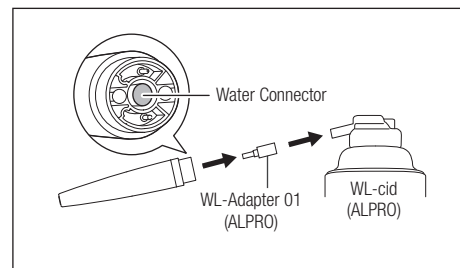2) Attach the WL-Adapter 01 (ALPRO) to the WL-cid (ALPRO) and connect the Water Connector on the Ultrasonic Handpiece to the end of the WL-Adapter 01 (ALPRO).

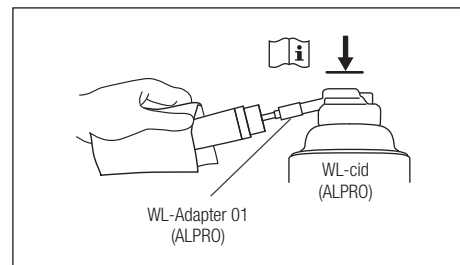
3) Spray the WL-cid (ALPRO) into the Water Connector on the Ultrasonic Handpiece while holding the Ultrasonic Handpiece and the WL-cid (ALPRO).


4) Remove the WL-Adapter 01 (ALPRO) and Ultrasonic Handpiece from the WL-cid (ALPRO).

5) Remove the Ultrasonic Handpiece from the WL-Adapter 01 (ALPRO), and then put the Ultrasonic Handpiece on the disinfected tray.

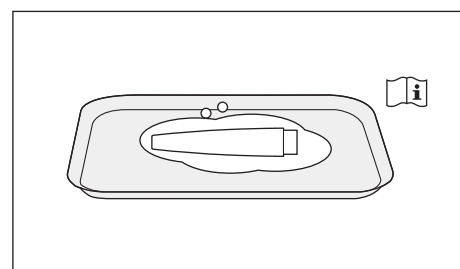6) Spray the WL-cid (ALPRO) to the Ultrasonic Handpiece surface, and then wait for the WL-cid (ALPRO) to take effect.

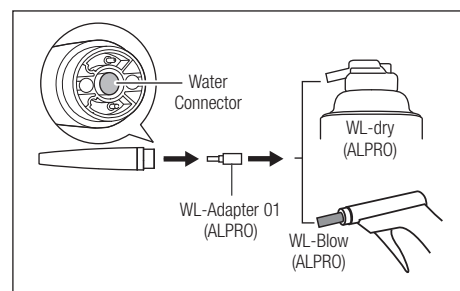
7) Attach the WL-Adapter 01 (ALPRO) to the WL-dry (ALPRO) or WL-Blow (ALPRO), and then attach the rear of the Ultrasonic Handpiece to the WL-Adapter 01 (ALPRO).


8) Drain disinfectant from the Ultrasonic Handpiece while holding the WL-dry (ALPRO) or WL-Blow (ALPRO) and the Ultrasonic Handpiece.

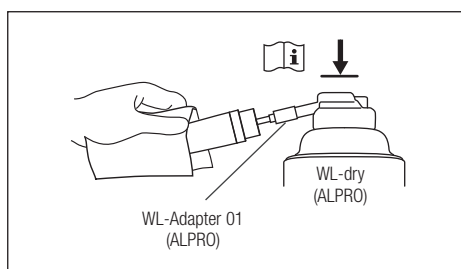· When using WL-dry (ALPRO):


· When using WL-Blow (ALPRO):

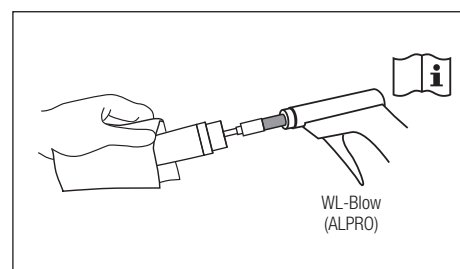9) Remove the WL-Adapter 01 (ALPRO) and Ultrasonic Handpiece from the WL-dry (ALPRO) or WL-Blow (ALPRO).

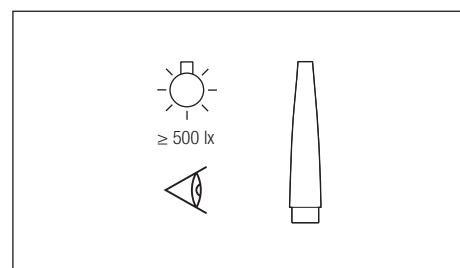10) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 1) until it is removed.

11) Proceed to the sterilization (refer to "9-2 Sterilization").

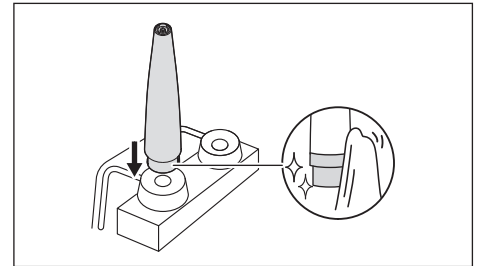
Auto

CAUTION

- Use a washer-disinfector that complies with DIN EN ISO 15883.
- Read the manual of your washer-disinfector.
- Since the glass rod is fragile, pay attention not to let the tip of the Ultrasonic Handpiece sustain an impact during cleaning.
- After the thermal disinfection and dry cycle are complete, take the product out of the washer-disinfector immediately (within an hour), and then dry it. Moisture remaining in the product causes internal corrosion or other problems.
- When using a washer-disinfector, use the adaptor compatible with the washer-disinfector (to be prepared by the user) to clean and disinfect the Ultrasonic Handpiece.
- For auto-cleaning and disinfection, follow the instructions in the manual of the manufacturer of the washer-disinfector.

■ Cleaning and disinfection (outside and inside)


1) Wipe the connection part of the Ultrasonic Handpiece using a lint-free cloth dampened with disinfectant alcohol, and then attach the Ultrasonic Handpiece to the adaptor of the washer-disinfector.


Adaptor compatible with the washer-disinfector (to be purchased by the user)

| Manufacturer | REF | Connection approx. | Color |
|--------------|-----------|--------------------|-------|
| Miele | ADS 2 | 16 mm | Green |
| MELAG | 63500 | 16 mm | Green |
| Getinge | 502887400 | 16 mm | Black |

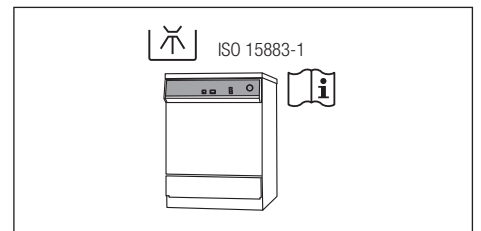
2) Perform the thermo disinfection following the manual of the washer-disinfector.

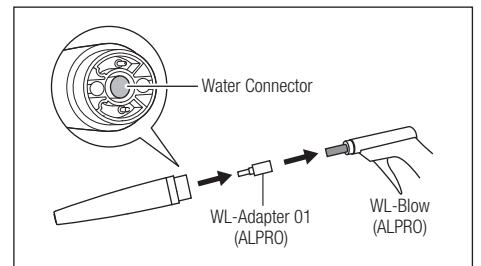
3) Take the Ultrasonic Handpiece out of the washer-disinfector.
 4) Attach the WL-Adapter 01 (ALPRO) to WL-Blow (ALPRO), and then attach the rear of the Ultrasonic Handpiece to the WL-Adapter (ALPRO).

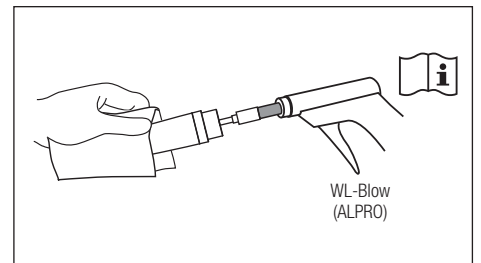5) Drain moisture from the Ultrasonic Handpiece while holding WL-Blow (ALPRO) and the Ultrasonic Handpiece.

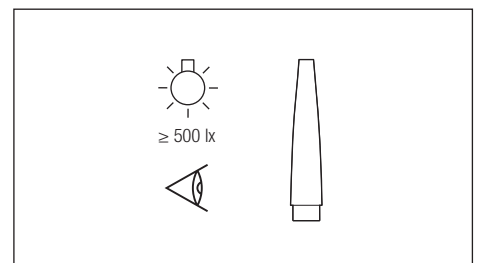
6) Remove the WL-Adapter 01 (ALPRO) and Ultrasonic Handpiece from the WL-Blow (ALPRO).
 7) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 2) until it is removed.

9-1-1-5 Glass rod

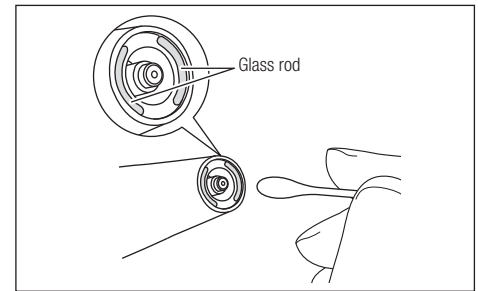If dust or debris adheres to the glass rod, carefully wipe it off using a cotton swab dampened with disinfectant alcohol.


CAUTION

- Do not use a needle or knife to clean the glass rod. Doing so damages the glass rod reducing its transmittance.


9-1-2 Handpiece Cord

Cleaning and disinfection

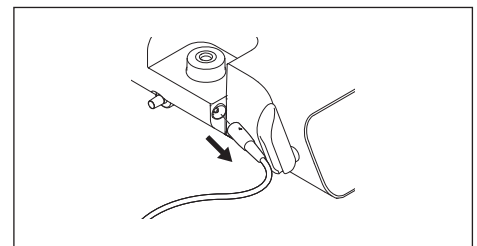
- 1) Disconnect the Handpiece Cord from the control unit.

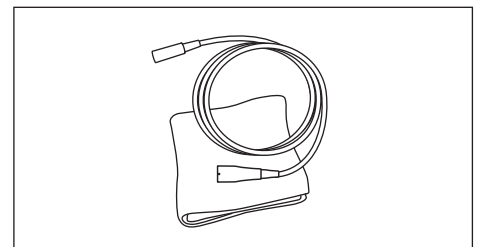
- 2) Wipe the Handpiece Cord surface using a lint-free cloth dampened with water.

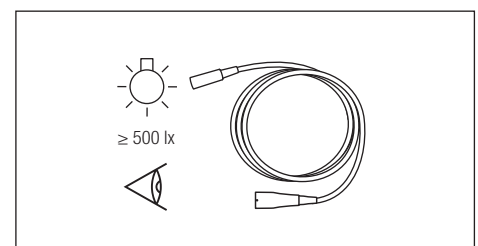- 3) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 2) until it is removed.

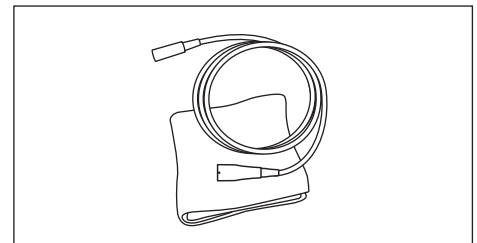
- 4) Wipe the Handpiece Cord surface using a lint-free cloth dampened with disinfectant alcohol.

- 5) Wipe the connection part using a lint-free cloth dampened with disinfectant alcohol.

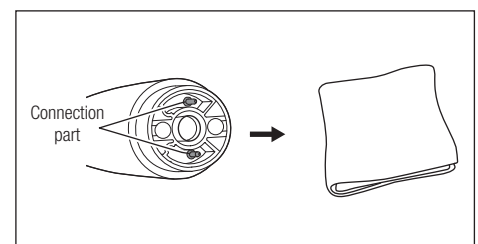If a cotton swab is too thick to clean the connection part, wrap wet tissue paper or the like around a thin stick and use it to wipe the connection part carefully.


CAUTION

- When cleaning the connection part, do not use a stick with a sharp point nor pick the terminal. Doing so may damage the contacts causing poor contact.


9-1-3 Irrigation Bottle

9-1-3-1 Cleaning and disinfection

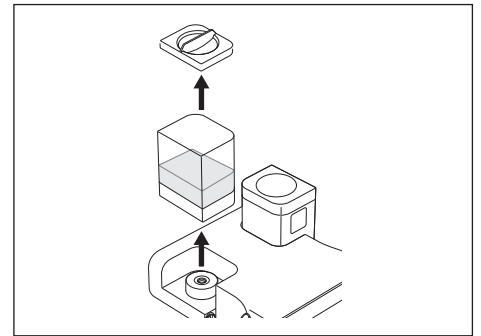
1) Remove the Irrigation Bottle from the Irrigation Bottle Base Connector then remove the Irrigation Bottle Cap from the Irrigation Bottle.

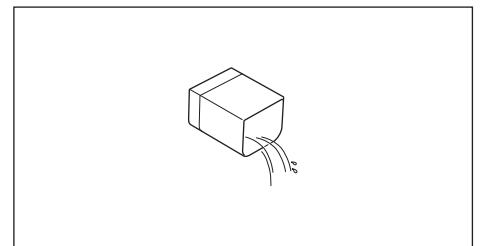
2) Drain the water or disinfectant remaining in the Irrigation Bottle.

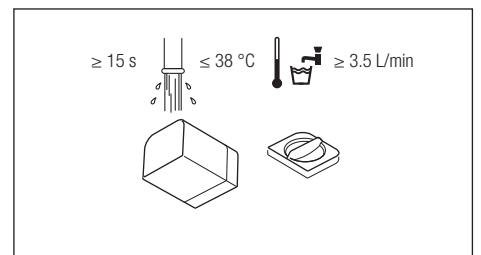3) Rinse the Irrigation Bottle and Irrigation Bottle Cap using running water.

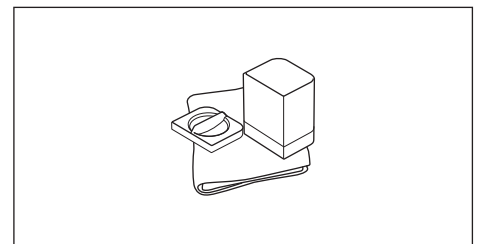4) Wipe the surfaces of the Irrigation Bottle and Irrigation Bottle Cap using a lint-free cloth dampened with disinfectant alcohol.


5) Attach the Irrigation Bottle to the Irrigation Bottle Base Connector.


9-1-4 Powder Handpiece

9-1-4-1 Removing nozzle tip

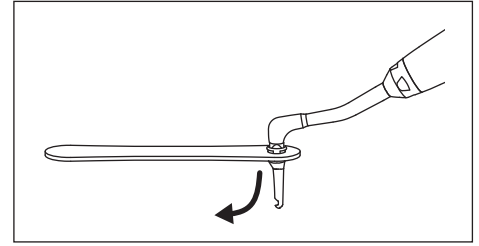1) Use the Perio Mate Nozzle Tip Remover to remove the Perio Mate Nozzle Tip.

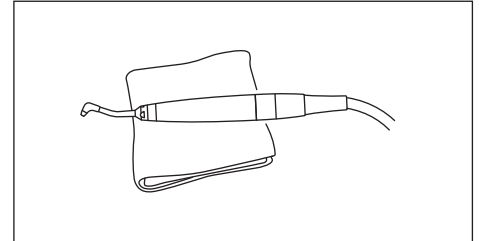2) Wipe off dirt from the Powder Handpiece surface using a lint-free cloth dampened with water.

WARNING

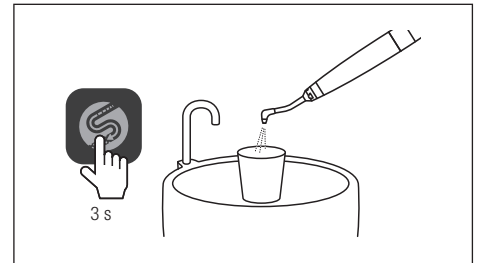
- A nozzle tip can be used only once. Do not reuse it. Doing so causes infection.
- Dispose of used Perio Mate Nozzle Tips as medical waste.

9-1-4-2 Auto-cleaning


CAUTION

- During auto-cleaning, water used for the cleaning comes out of the tip of the Powder Handpiece. Prepare a container such as a cup to catch the water.
- Before starting auto-cleaning, touch the Irrigation Volume Keys (▲) to adjust the Water Level to L5 (refer to "11-3 Basic Water Level Setting").

1) Point the tip of the Powder Handpiece at a container for catching water, and then touch  for three seconds.
The auto-cleaning starts. (Cleaning time is 45 seconds.)

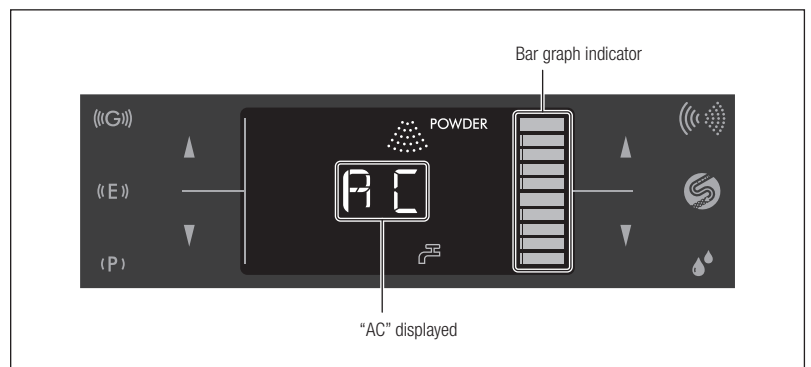
2) During the auto-cleaning, "AC" is shown on the LCD.
The bar graph indicator shows the progress.

* To cancel auto-cleaning, touch  again.


When auto-cleaning is complete, the control unit returns to the state before the auto-cleaning cycle began.

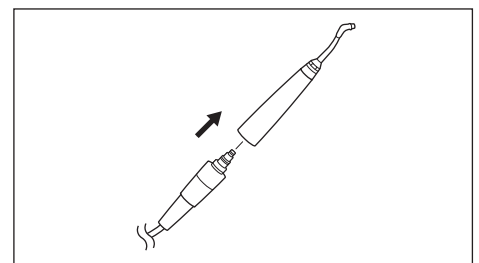3) Disconnect the Powder Handpiece from the Handpiece Hose.


9-1-4-3 Removing powder

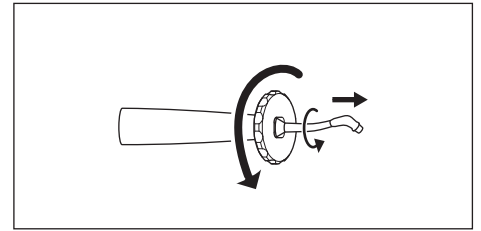1) Remove the Nozzle from the Powder Handpiece using the Ring Wrench (accessory).

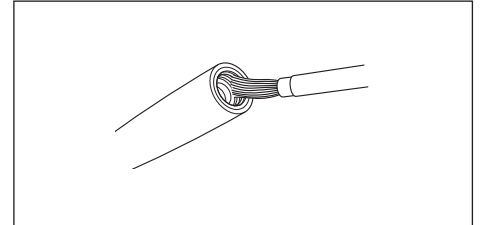2) Remove the powder adhering to the inside of the connection part of the Powder Handpiece using the Cleaning Brush (accessory).


WARNING


- Be sure to clean the inside of the Powder Handpiece after use. Powder remaining in the Powder Handpiece may harden causing clogging.

9-1-4-4 Pre-cleaning

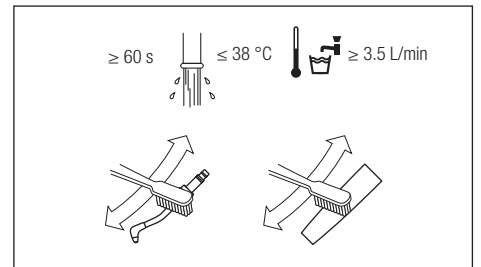
1) Clean the Powder Handpiece and Nozzle using running water and a soft bristle brush.

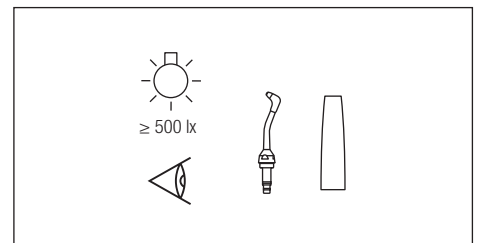2) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 1) until it is removed.

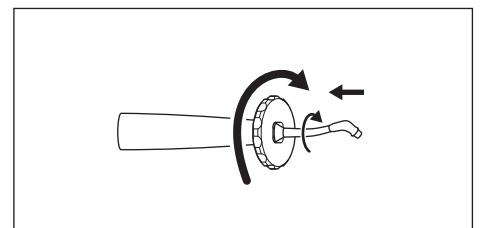3) Attach the Nozzle to the Powder Handpiece.

CAUTION

- Perform ultrasonic cleaning at least once every week to prevent powder from clogging. Perform ultrasonic cleaning before performing auto-cleaning or after applying the remedy described in "14 Troubleshooting" ("9-3 Ultrasonic cleaning (weekly)").
- Be sure to use the supplied Ring Wrench to attach and remove Nozzles. When attaching or removing a Nozzle, do not hold its distal end. Doing so may deform the Nozzle.
- Do not use ultrasonic cleaning to clean Powder Handpieces.

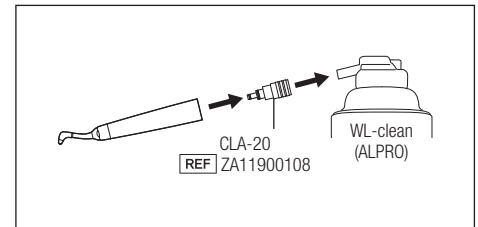
CAUTION

- If the O-ring on the CLA-20 is deteriorated or damaged, replace it with a new one before use.
- Be sure to do pre-cleaning before cleaning.
- After disinfection, be sure to sterilize the Powder Handpiece (refer to “9-2 Sterilization”).
- Be sure to carry out disinfection within two hours after cleaning.
- When using any spray (ALPRO), be sure to cover the Powder Handpiece head with a cloth to prevent detergent and disinfection agent from scattering. We recommend that you use the spray mist absorber (Y900084).
- When using ALPRO products, refer to the manual of the manufacturer.


Manual

■ Cleaning

1) Attach the CLA-20 (REF ZA11900108) to the WL-clean (ALPRO), and then attach the Powder Handpiece to the CLA-20.

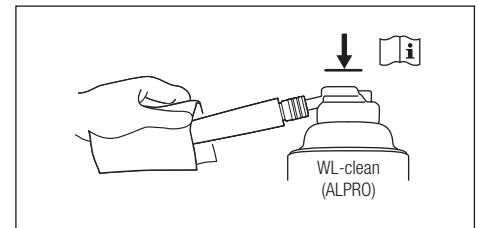
2) Spray the WL-clean (ALPRO) while holding the WL-clean (ALPRO) the Powder Handpiece.


3) Remove the CLA-20 and Powder Handpiece from WL-clean (ALPRO).

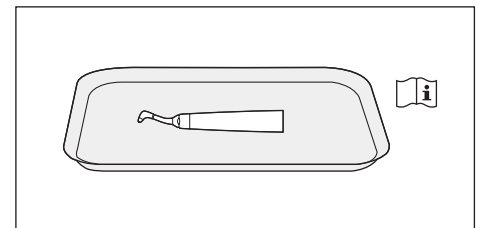4) Wait for the WL-clean (ALPRO) to take effect.


■ Disinfection

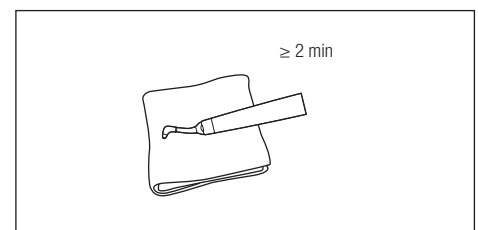
1) Wipe the Powder Handpiece surface using a lint-free cloth dampened with disinfectant alcohol.

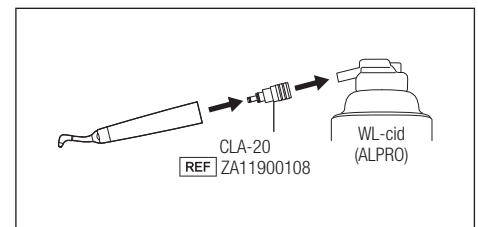
2) Attach the CLA-20 (REF ZA11900108) to WL-cid (ALPRO), and then attach the Powder Handpiece to the CLA-20.

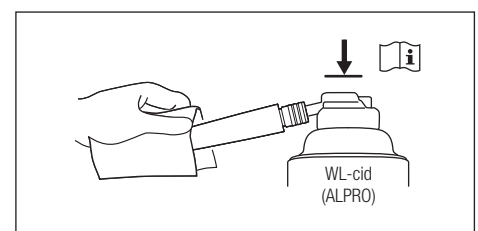
3) Spray WL-cid (ALPRO) to the Powder Handpiece while holding WL-cid (ALPRO) and the Powder Handpiece.

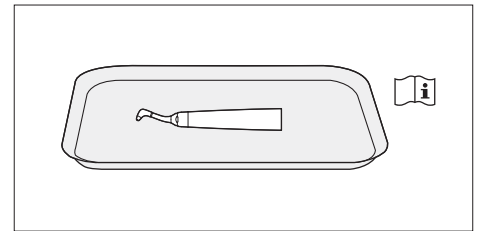- 4) Remove the CLA-20 and Powder Handpiece from WL-cid (ALPRO).
- 5) Wait for the WL-cid (ALPRO) to take effect.

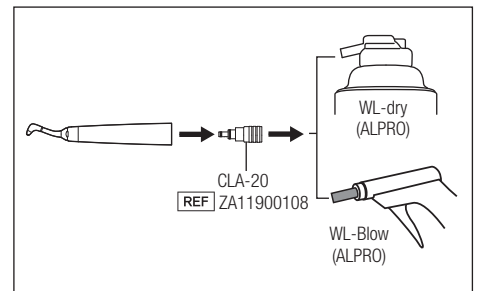
- 6) Attach the CLA-20 to WL-dry (ALPRO) or WL-Blow (ALPRO), and then attach the Powder Handpiece to the CLA-20.


- 7) Drain disinfectant from the Powder Handpiece while holding WL-dry (ALPRO) or WL-Blow (ALPRO) and the Powder Handpiece.

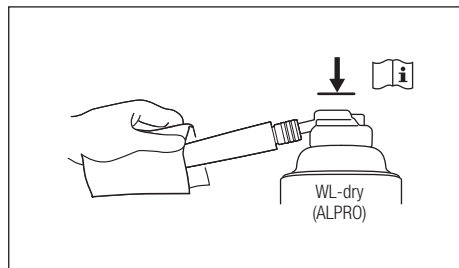
· When using WL-dry (ALPRO):

· When using WL-Blow (ALPRO):

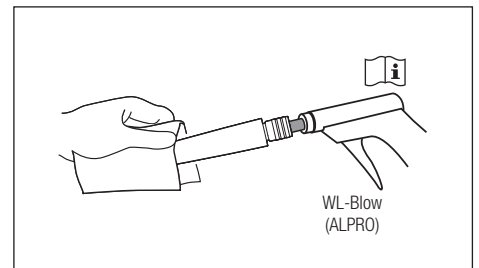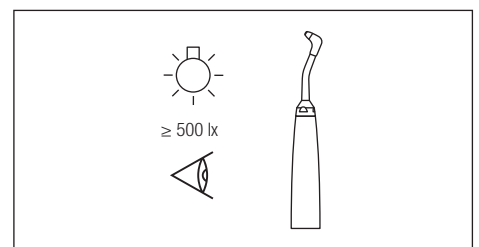- 8) Remove the Powder Handpiece from WL-dry (ALPRO) or WL-Blow (ALPRO).
- 9) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 1) until there is no dirt.

- 10) Proceed to the sterilization (refer to "9-2 Sterilization").

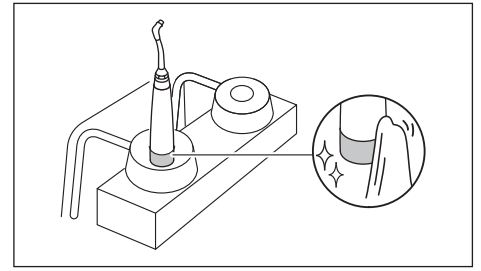
Auto

CAUTION

- Use a washer-disinfector that complies with ISO 15883.
- Read the manual of your washer-disinfector.
- Sufficiently dry the Powder Handpiece after cleaning with a washer-disinfector. Remaining moisture causes internal corrosion and poor lubrication.
- After the thermal disinfection and dry cycle are complete, take the product out of the washer-disinfector immediately (within an hour), and then dry it. Moisture remaining in the product causes internal corrosion and poor lubrication.
- When using a washer-disinfector, use the adaptor compatible with the washer-disinfector (to be prepared by the user) to clean and disinfect the Powder Handpiece.
- For the auto-cleaning and disinfection, follow the instructions in the manual of the manufacturer of the washer-disinfector.

■ Cleaning and disinfection


- 1) Wipe the connection part of the Powder Handpiece using a lint-free cloth dampened with disinfectant alcohol, and then attach the Powder Handpiece to the adaptor of the washer-disinfector.


Adaptor compatible with the washer-disinfector (to be purchased by the user)

| Manufacturer | REF | Connection approx. | Color |
|--------------|-----------|--------------------|-------|
| Miele | ADS 2 | 16 mm | Green |
| MELAG | 63500 | 16 mm | Green |
| Getinge | 502887400 | 16 mm | Black |

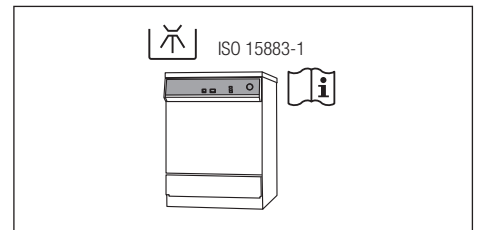
- 2) Perform the thermo disinfection following the manual of the washer-disinfector.

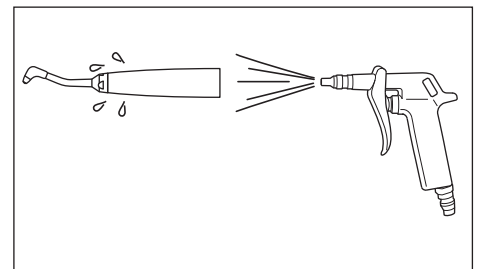- 3) Remove the moisture from the surface and inside of the Powder Handpiece using an air blow gun or similar tool.

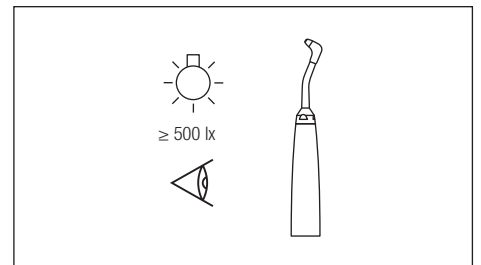- 4) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 2) until there is no dirt.


- 5) Proceed to the sterilization (refer to "9-2 Sterilization").

9-1-5 Chamber and Handpiece Hose


Perform the following maintenance at the end of the day.

9-1-5-1 Removing powder

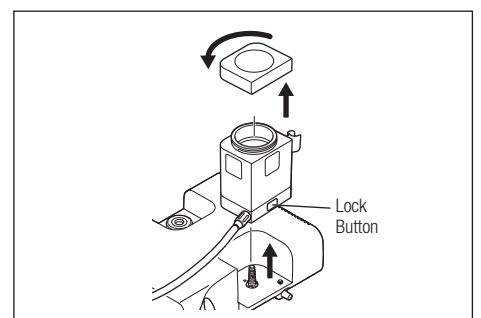- 1) Release the residual pressure from the Chamber (refer to "8-6-4 Releasing residual pressure").
- 2) Push the Lock Button, and then remove the Chamber from the Chamber Base Connector.
Remove the Chamber Cap.

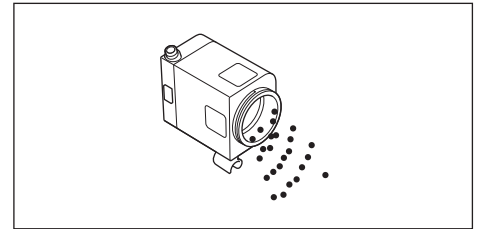
3) Dispose of the powder in the Chamber (refer to “22 Disposing product”).


4) Close the Chamber Cap firmly, and then insert the Chamber straight into the Chamber Base Connector.

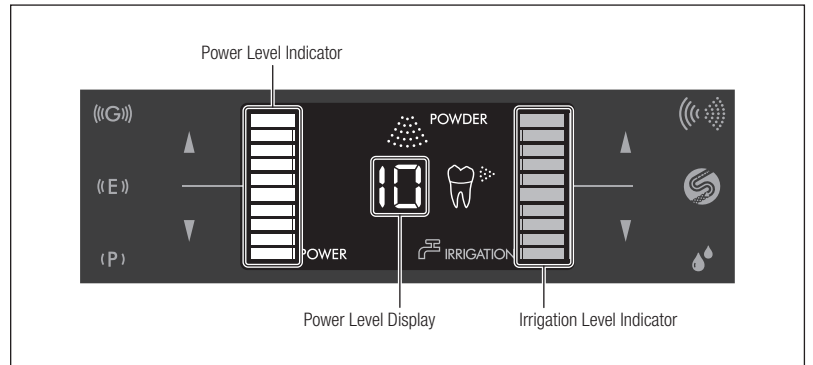5) Set the power level and the irrigation level to “10” and the maximum, respectively.

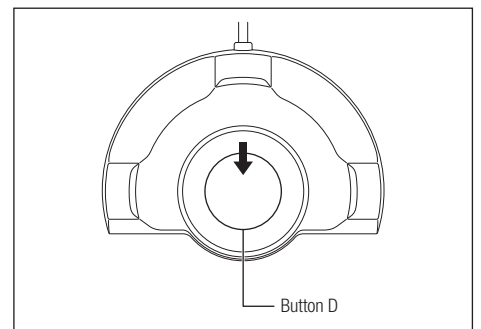
6) Point the Handpiece Hose at a spittoon or sink and depress the button D of the Foot Control to run 5 times for more than a second.


7) Perform auto-cleaning to drain the remaining powder and water from the Handpiece Hose (refer to 9-1-4-2 Auto-cleaning).

9-1-5-2 Cleaning and disinfection

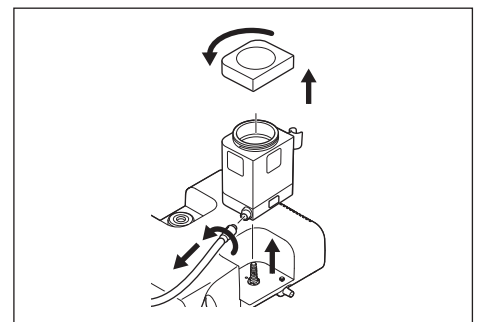
- 1) Release the residual pressure from the Chamber (refer to “8-6-4 Releasing residual pressure”).
- 2) Push the Lock Button, and then remove the Chamber from the Chamber Base Connector.
Remove the Chamber Cap and disconnect the Handpiece Hose.
* If the Chamber cannot be removed even though you pushed the Lock Button, release the Lock Button (refer to “9-1-5-3 Releasing Lock Button”).

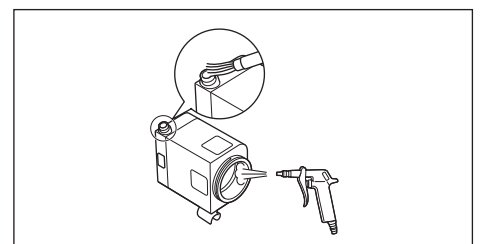3) Remove the powder remaining in the Chamber and Handpiece Hose Connector Socket using an air blow gun or similar tool.

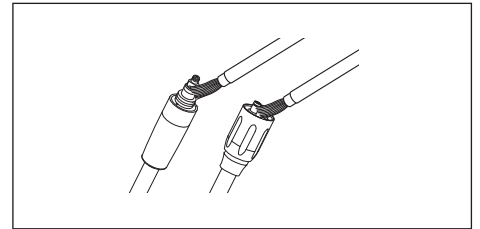
4) Remove the remaining powder from both ends of the Handpiece Hose using the Cleaning Brush (accessory).


5) Make sure that no powder remains on the threads of the Chamber Cap. If any powder remains, clean it off. (Refer to “Cleaning Chamber Cap”)

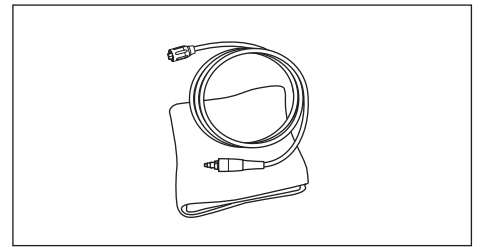
6) Wipe the Handpiece Hose surface using a lint-free cloth dampened with water.

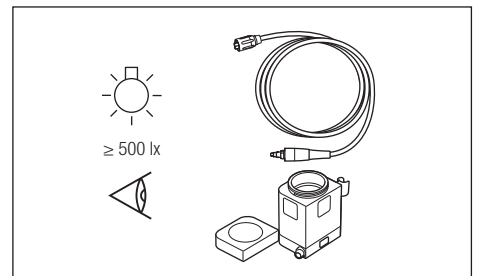7) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat from step 3) until it is removed.

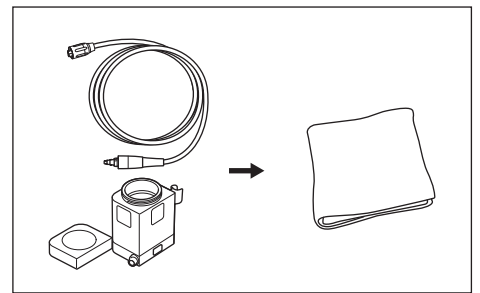8) Wipe the surfaces of the Chamber, Chamber Cap, and Handpiece Hose using a lint-free cloth dampened with disinfectant alcohol.

9) Proceed to the storage (refer to “9-4 Storage”) or insert the Chamber straight into the Chamber Base Connector.

WARNING

- Securely attach the Chamber Cap. Otherwise, the Chamber Cap may be blown off during use, which may lead to damage to the device or an injury.

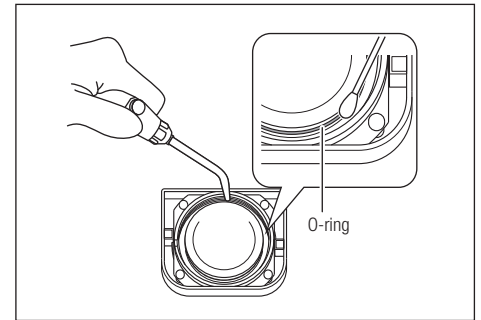
CAUTION

- If powder adheres to the threads of the Chamber and/or the Chamber Cap, the cap will become difficult to close. Cleaning the Chamber and Chamber Cap makes it easy to open and close the cap (refer to “Cleaning Chamber Cap”).
- If the Chamber malfunctions, be sure to request your authorized NSK dealer to repair it.
- After using, remove the powder adhering to the inside of the Chamber completely and clean the Chamber before storing it.
- Do not wash the Chamber with water. Doing so solidifies powder causing clogging. The air pressure in the Chamber may be high making it difficult to open the cap for awhile just after use. In such a case, wait awhile or release the residual pressure before opening the cap (refer to “8-6-4 Releasing residual pressure”).
- Before connecting the Handpiece Hose, Chamber and Handpiece, or the Chamber and Control Unit, be sure to blow off all water from the connections of the Hose and the Chamber with dry air. Failure to do so may result in moisture entering the air supply, causing the cleaning powder inside the product to solidify.

Cleaning Chamber Cap

If powder adheres to the threads of the Chamber and/or the Chamber Cap, the Chamber Cap will become difficult to close. Cleaning the Chamber and Chamber Cap following the procedure shown below makes it easy to open and close the cap.

- 1) Remove the Chamber Cap.
- 2) Clean the inside of the Chamber, threads, and the fringe of the O-Ring of the Chamber Cap using a brush or dry cotton swab.


- 3) Remove the adhering powder by blowing air.
- 4) Proceed to storage (refer to "9-4 Storage") or insert the Chamber straight into the Chamber Base Connector.

9-1-5-3 Releasing Lock Button

If the Lock Button cannot be pushed making it impossible to remove the Chamber, follow the procedure shown below to release the Lock Button and remove the Chamber.

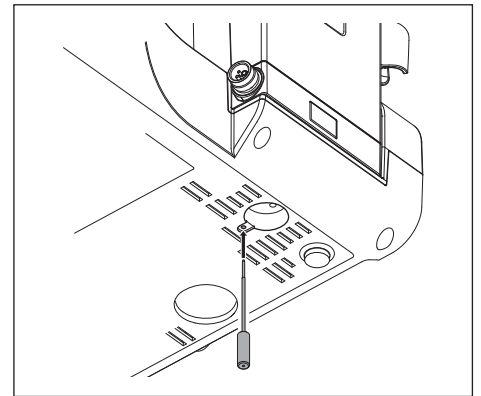- 1) Insert the Unlocking tool (accessory) fully into the hole on the bottom of the control unit to release the Lock Button.

- 2) Pull out the Unlocking tool.
 - 3) Push the Lock Button, and then remove the Chamber.
- If the procedure shown above fails to release the Lock Button, contact your authorized NSK dealer.

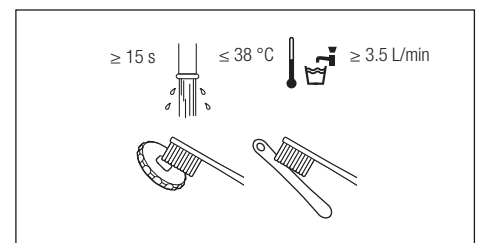
- Do not invert the control unit when releasing the Lock Button. Inverting the Chamber could lead to powder clogging.
- Do not twist the Unlocking tool when inserting it.


9-1-6 Ring Wrench and Perio Mate Nozzle Tip Remover

9-1-6-1 Cleaning and disinfection


- 1) Clean the Ring Wrench and Perio Mate Nozzle Tip Remover for 15 seconds or more using running water and a soft bristle brush.

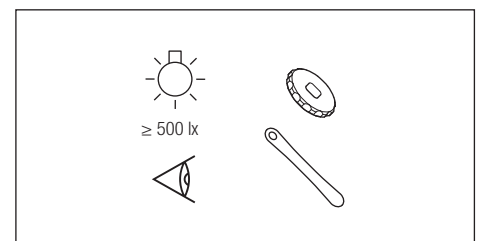- 2) Check for dirt under appropriate illumination (500 lx or more).

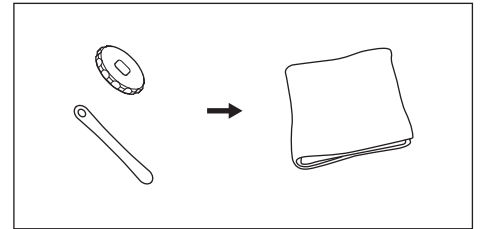- 3) Wipe the Ring Wrench and Perio Mate Nozzle Tip Remover using a lint-free cloth dampened with disinfectant alcohol.


- 4) Proceed to sterilization (refer to “9-2 Sterilization”).

9-1-7 Control Unit and Foot Control

9-1-7-1 Cleaning

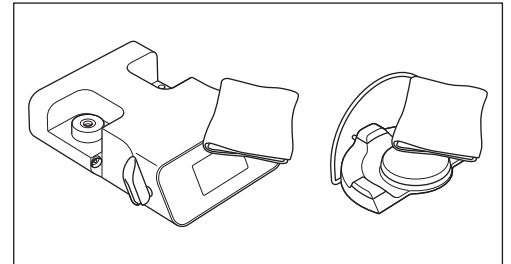- 1) Turn off the power.
- 2) Wipe off dirt from the surfaces of the Control Unit and Foot Control using a lint-free cloth dampened with water.
- 3) Wipe the surfaces of the Control Unit and Foot Control using a lint-free cloth dampened with disinfectant alcohol.

- 4) Proceed to storage (refer to “9-4 Storage”).

CAUTION

- Do not use solvents such as electrolyzed-oxidizing water (strong acidic water or super acidic water), strong acidic or alkaline chemical, chlorine-containing solvent, benzene, and thinner when cleaning, immersing, or wiping.

9-2 Sterilization


The effectiveness of the following sterilization procedure has been verified complying with ISO17665.

Sterilizable items:

Ultrasonic Handpiece, Powder Handpiece, Ring Wrench, Perio Mate Nozzle Tip Remover

- 1) Put the items into a sterilization pouch that complies with ISO 11607-1, and then seal the pouch.
- 2) Sterilize them with the following conditions.

| Type | Gravity displacement (Class N) | Pre-vacuum (Class B) |
|--------------------|--------------------------------|----------------------|
| Temperature | 132 °C | 134 °C |
| Sterilization time | ≥15 min | 3-18 min |

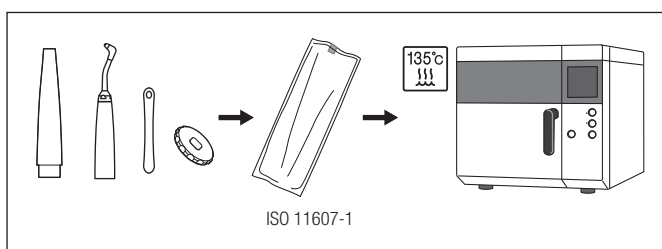- 3) Proceed to storage (refer to “9-4 Storage”).

CAUTION

- Always obey the local directives, standards, guidelines, etc., as to the reprocessing of equipment.
- Use a sterilization pouch that complies with ISO 11607- 1.
- Do not place other instruments together with the items of this product in a sterilization pouch to perform the autoclave sterilization. Doing so discolors the product surface and affects internal parts. In addition, pay attention not to let chemicals enter the product during the autoclave process.
- Be sure to use sterilizers that can perform sterilization up to 135 °C.
Do not use a sterilizer that exceeds 135 °C during the dry cycle. It may cause the product to fail. For details, consult the manufacturer of the sterilizer.
- Store the product in a location where there is no risk of adverse effects of the atmospheric pressure, temperature, humidity, ventilation, sunlight, dust, salt, and sulfuric atmosphere.
- Do not heat or cool the product rapidly during the sterilization. Rapid changes in the temperature deteriorate the product.
- Do not touch the product just after the sterilization because it is hot.
- As to this product, the effectiveness of sterilization methods (such as the plasma sterilization and EOG sterilization) other than the steam sterilization is not verified.
- After the sterilization is complete, take the product out of the sterilizer within an hour. Failure to do so may cause corrosion.
- Do not sterilize the parts other than the sterilizable items.

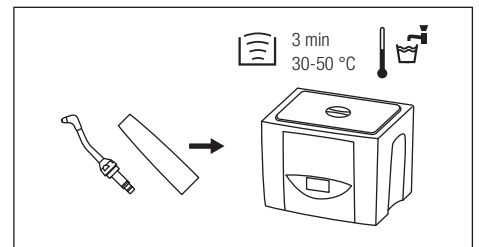
NOTICE

- We recommend the class B sterilizer stipulated in EN13060.
For details, refer to the operation manual issued by the manufacturer of the sterilizer.
- Although the handpiece may discolor due to the heat after repetitions of the autoclave sterilization, it is due to the characteristics of the product and does not indicate any problem in the quality of the product.


9-3 Ultrasonic cleaning (weekly)

Be sure to perform the cleaning and disinfection before ultrasonic cleaning (refer to “9-1-4 Powder Handpiece”).


- 1) Clean the Powder Handpiece and Nozzle using ultrasonic cleaning.

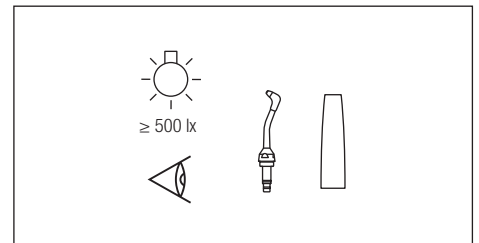
- 2) Check for dirt under appropriate illumination (500 lx or more). If there is dirt, repeat step 1) until the dirt is removed.

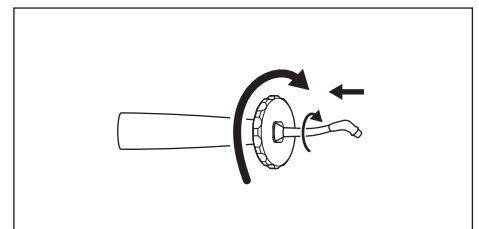- 3) Attach the Nozzle to the Powder Handpiece using the Ring Wrench (accessory).

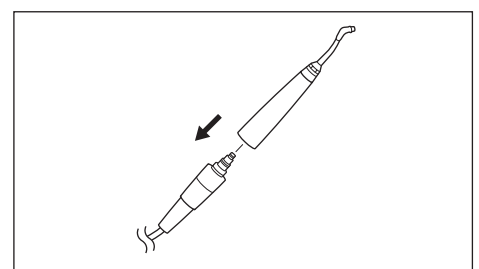- 4) Insert the Powder Handpiece straight into the Handpiece Hose Plug.

- 5) Perform the auto-cleaning and sterilization (refer to “9-1-4-2 Auto-cleaning” and “9-2 Sterilization”).

- CAUTION**
- After ultrasonic cleaning, perform auto-cleaning to remove internal moisture, and then dry the product sufficiently.
 - After auto-cleaning, be sure to perform sterilization.
 - Do not use ultrasonic cleaning to clean Ultrasonic Handpieces. It causes failure.

9-4 Storage

Store the product in an environment where the product is kept disinfected and sterilized until the next use.

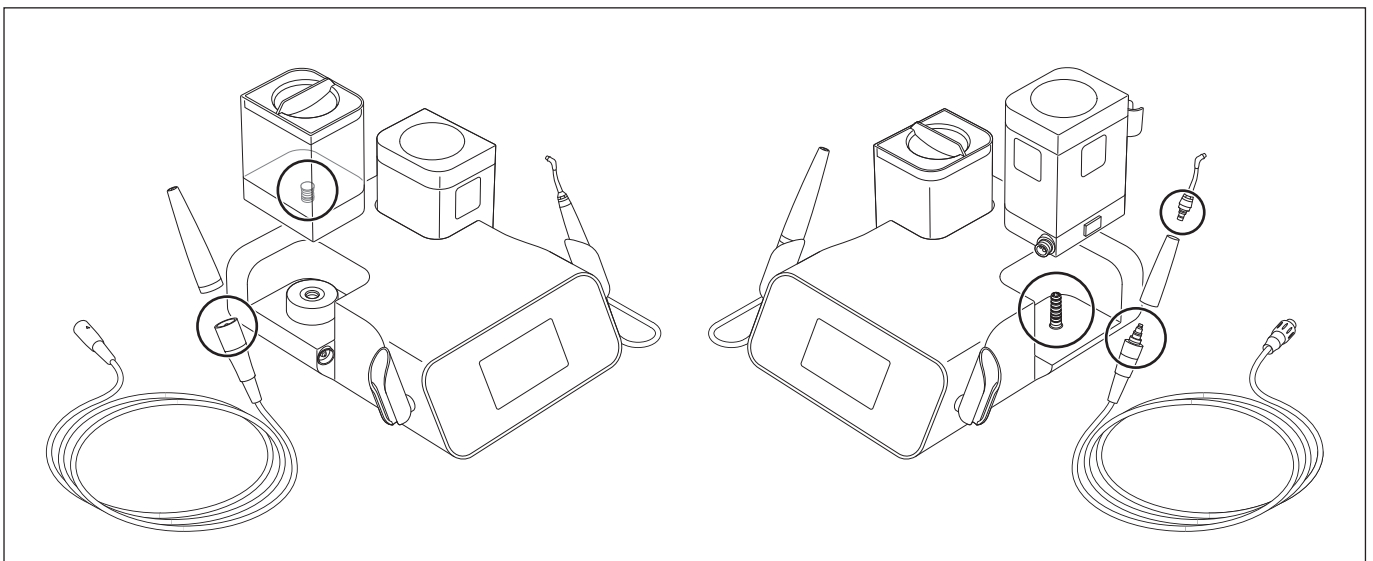
- CAUTION**
- Store the product in a location where there is no risk of adverse effects of the atmospheric pressure, temperature, ventilation, sunlight, dust, salt, and sulfuric atmosphere.

10 Care and Maintenance

10-1 Maintenance of the O-rings

Perform maintenance to reduce wear of the O-rings on the target parts and prevent leakage of water or air.


* For replacing the O-ring of the Chamber Cap, contact your Authorized NSK Dealer.

10-1-1 Handpiece Cord

- 1) Remove the Handpiece Cord from the Control Unit.
 - 2) Remove the O-ring to be replaced using a needle, etc.
 - 3) Place the new O-ring in the groove from where the old O-ring was removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.

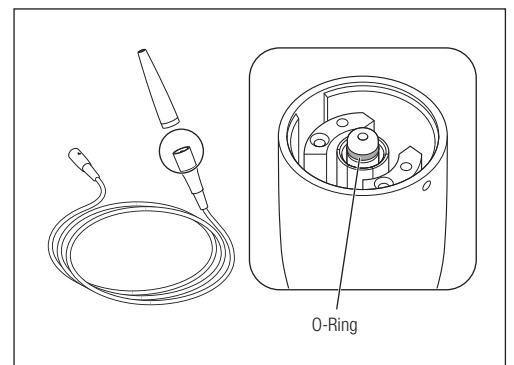REF Y1003546


10-1-2 Handpiece Hose (for Prophy use and Perio use)

- 1) Remove the Hose from the Powder Handpiece and Chamber.
 - 2) Remove the O-rings from the Handpiece Hose Plug using a needle, etc. If there is powder in the O-ring grooves, clean the groove before inserting the new O-ring.
 - 3) Place the new O-rings in the grooves from where the old O-rings were removed.
 - 4) Apply a drop of the supplied turbine oil and rub it in with your finger.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.

REF Y1003500

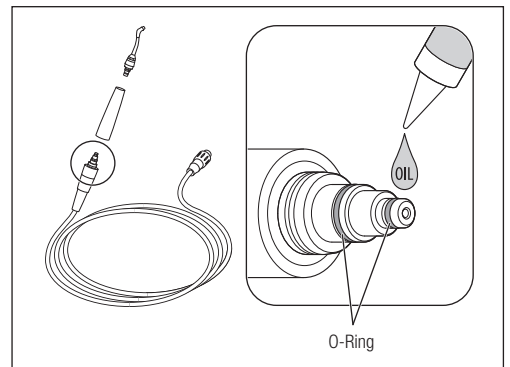
- Failure to lubricate the O-ring section on the Handpiece side may cause damage to the Joint section, resulting in difficulty in rotating the Handpiece connection. It may also cause water leakage.

10-1-3 Nozzle (for Prophy use and Perio use)

- 1) Pass through the supplied Ring Wrench from the Nozzle end, loosen the Nozzle, and remove the Nozzle from the Handpiece.
 - 2) Remove the O-rings from the root of the Nozzle using a needle, etc. If there is powder in the O-ring grooves, clean the groove before inserting the new O-ring.
 - 3) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.

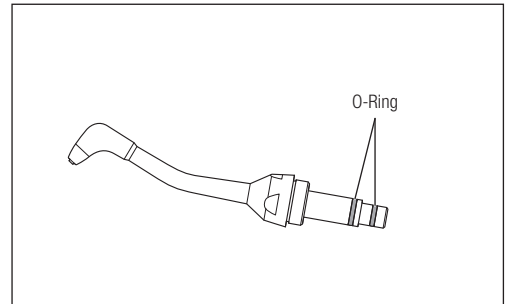REF Y1003501


10-1-4 Irrigation Bottle

- 1) Remove the O-rings from the Irrigation Bottle Joint using a needle, etc.
 - 2) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.

REF Y1003544

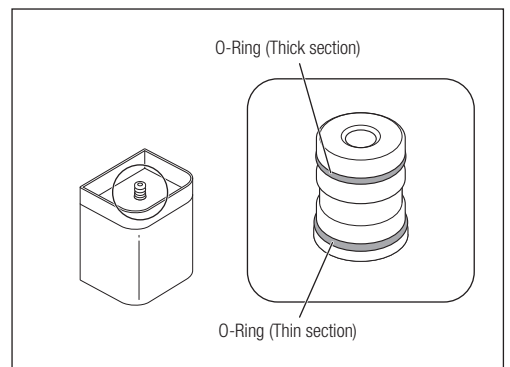- The O-rings are of different thicknesses. Be sure to place them in the correct groove.


10-1-5 Chamber Base Connector

- 1) Remove the O-rings from the Chamber Base Connector using a needle, etc.
 - 2) Place the new O-rings in the grooves from where the old O-rings were removed.
- * Refer to "21 Spare Parts List" for details about replacement O-rings.

REF Y1003435

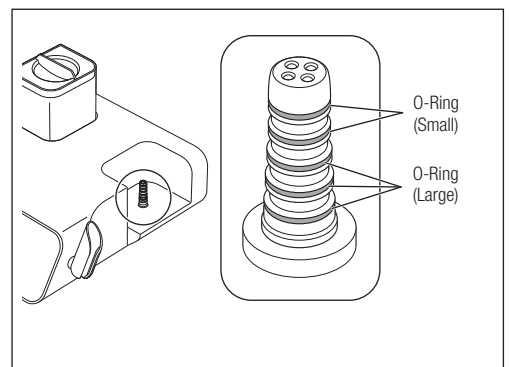
- Since the size of the O-Rings differ, make sure to replace them in the correct location.


10-2 Changing the Irrigation Pump

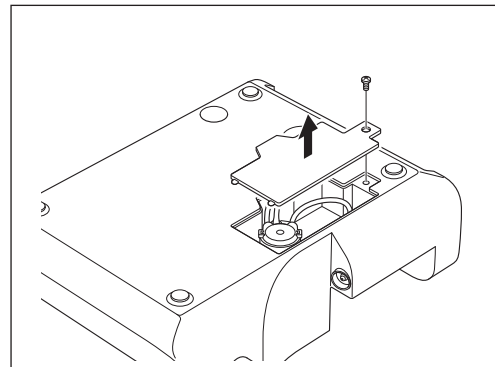
- 1) Remove the Irrigation Bottle, the Chamber, Power Cord, Handpiece Cord and Foot Control from the Control Unit.
- 2) Turn over the Control Unit. Use a cross-head screwdriver to remove the screw on the Irrigation Pump Cover and remove the Cover.

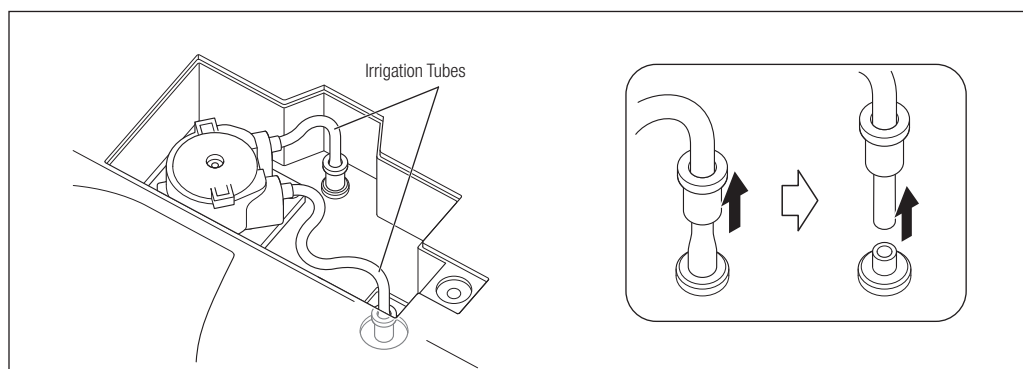
- 3) Remove two Irrigation Tubes from the Control Unit.

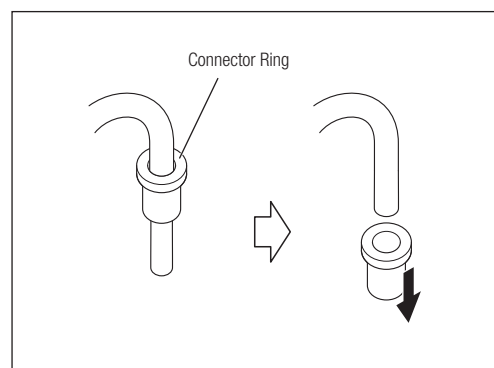
- 4) Remove the Connector Ring from the Irrigation Tube. Do not dispose of it. You can use the Ring on the replacement Irrigation Pump.

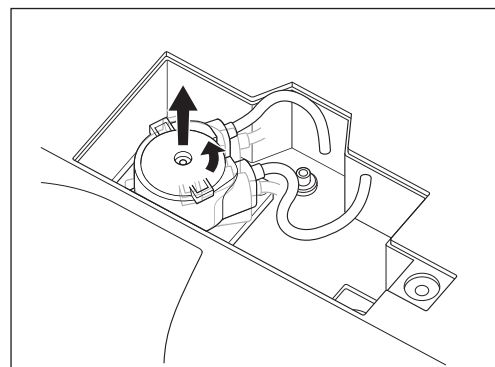
- 5) Turn the Irrigation Pump counterclockwise until it clicks, and then pull it out.

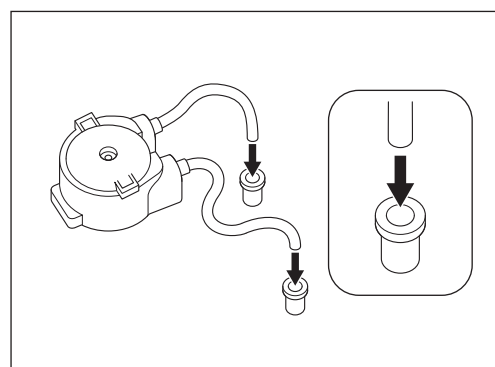
- 6) Mount the Connector Rings in the Irrigation Tubes of the new Irrigation Pump (option). Make sure the Ring is oriented correctly.

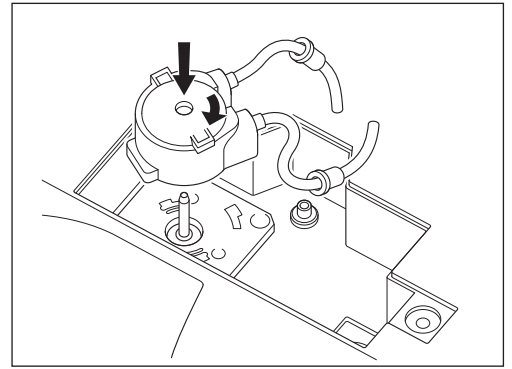7) Align the new Irrigation Pump (option) with the Drive Shaft. Turn clockwise until it clicks.

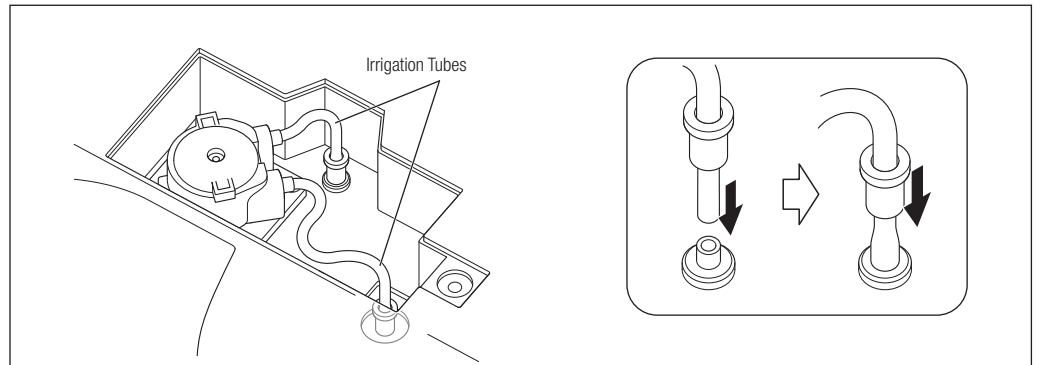8) Mount the Irrigation Tube. Press the Connector Ring firmly into the Control Unit until it is fully pressed on.


9) Align the Irrigation Pump Cover Hook and hole on the Control Unit. Mount the Cover.

* For details on the Optional Irrigation Pump, refer to "21 Spare Parts List."

CAUTION

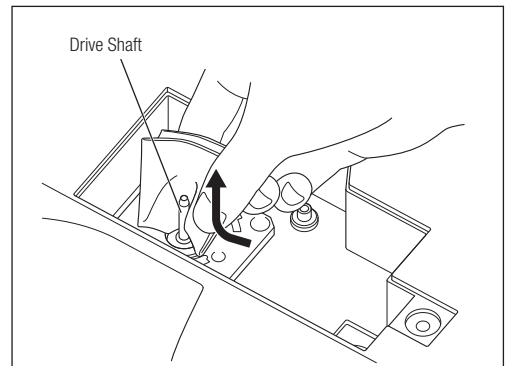- Before replacing the Irrigation Pump, wipe off excess water from the pump and Drive Shaft. Wet drive shaft and rollers can be slippery and cause improper operation.
- Wipe dirt and water off the Drive Shaft from the bottom up.
- Insert the replacement Irrigation Pump straight into the Drive Shaft in a slow and gentle manner to prevent damaging the pump rollers.
- After replacing the Irrigation Pump, run this product for about 10 seconds with Irrigation Mode set to Bottle and Irrigation Volume set to maximum so the Irrigation Tube adapts to the new Pump.
- Ensure there are no kinks or twists in the Irrigation tube. If tube is set incorrectly, Irrigation Water may not be supplied.
- Make sure that the Tube is not caught under the Irrigation Pump Cover when closing the Irrigation Pump Cover.
- Only use the dedicated Irrigation Pump for Varios Combi Pro. Other pumps cannot be used.

NOTICE

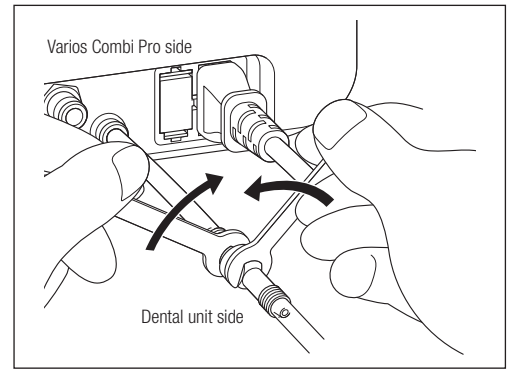
- Clean the Drive Shaft periodically with a lint-free cloth dampened with disinfectant alcohol. Debris on Drive Shaft may cause incorrect pump operation.
- The pump is a wear item. If the irrigation volume decreases markedly, replace the pump.

10-3 Changing the Water Filter

If you use Tap Water, change the Water Filter once a year.

If you feel that the water volume has dropped, change the Water Filter before a year has elapsed.

- 1) Close the water valve of the dental unit.
- 2) Place the Control Unit on a cloth, etc. in case water leaks.
- 3) Use two spanners (5x8) and turn in the directions shown in the figure.
- 4) Remove the Water Filter Case and take out the Water Filter.


- 5) Place the new Water Filter with the O-ring on the Varios Combi Pro side and reassemble by following the disassembly steps in reverse.

* Refer to "21 Spare Parts List" for details about the optional Water Filter.

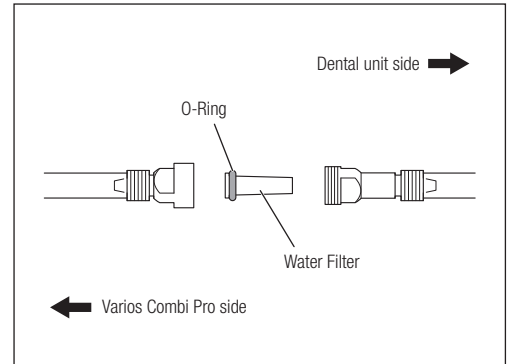
- Make sure that the Water Filter is placed in the correct orientation. Otherwise the Filter will not work effectively.

10-4 Checking the Air Filter

When symptoms such as reduced Power Level or frequent clogging of powder are observed, release water by pressing the tip of the Air Filter at the bottom of the Control Unit.

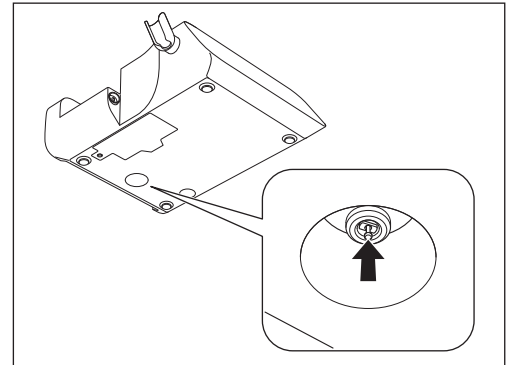
If sufficient Power Level cannot be obtained as a result of prolonged use, the VCP Element may have clogged. If this happens, contact your Authorized NSK Dealer.

10-5 Replacing the fuse

- 1) Turn OFF the Power Switch.
- 2) Unplug the AC power cord from the wall outlet.
- 3) Remove the Fuse Holder located at the back of the Control Unit while keeping the nails at both ends pushed inward.
- 4) Replace the fuse with a new one.
- 5) Push the Fuse Holder back into the place.

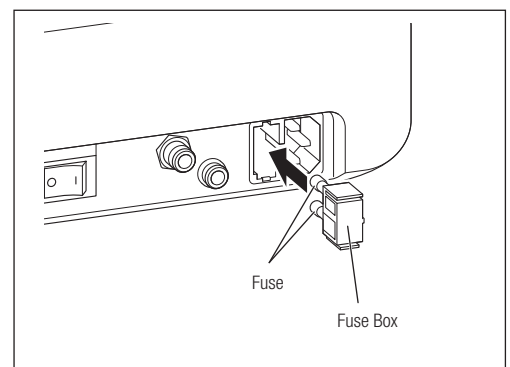* Refer to "21 Spare Parts List" for details about replacement Fuses.

- Only use NSK recommended fuses. Failure may occur if other fuses are used.
- If a fuse blows but you cannot establish the reason, contact your Authorized NSK Dealer.

NOTICE • If the Fuse Box does not come out easily, use a hard-tipped implement to push the nails while pulling the Fuse Box out.

11 Other Functions

11-1 Initialized Program (Factory Setting)

The setting value for each of the modes are stored in this product. These values can be initialized to the factory settings.


- 1) Turn the POWER switch ON while depressing the Foot Control button B.
- 2) "Clr" is displayed on the LCD Panel. While "Clr" is displayed, release the Foot Control button B and then depress the button D to start initializing.
- 3) "Fin" is displayed on the LCD Panel when the initialization is complete. An alarm sound is heard and the product starts in the normal mode. (Refer to the table below for the initial settings.)

| | Ultrasonic Mode | Powder Mode | |
|---------------------------|-----------------|---------------------------|------------|
| | | Prophy Mode | Perio Mode |
| Ultrasonic Mode selection | Perio | - | |
| Power Level | 1 | 1 | 1 |
| Irrigation Mode | Bottle | - (fixed to tap water) | |
| Irrigation Volume | 10 | 10 | 10 |

ULTRASONIC

11-2 Using the Tip Holder (for Ultrasonic Tip, option)

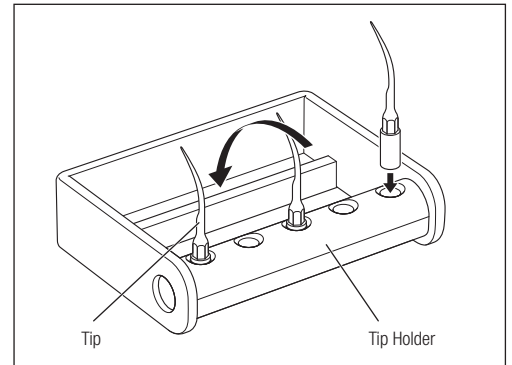
- Use the Tip Holder to store the Tips after removing them from the Handpiece.
- The Tip Holder is autoclavable. Up to 5 Tips can be sterilized at the same time. Rotate the Holder in the direction of the arrow so the Tips are set in the frame for sterilization.

11-3 Basic Water Level Setting

Basic Water Level can be adjusted depending on the supply water pressure in the usage environment. If the Irrigation Volume is too large or too small even when adjusted by the Irrigation Volume Keys, follow the procedure below to adjust the Basic Water Level.

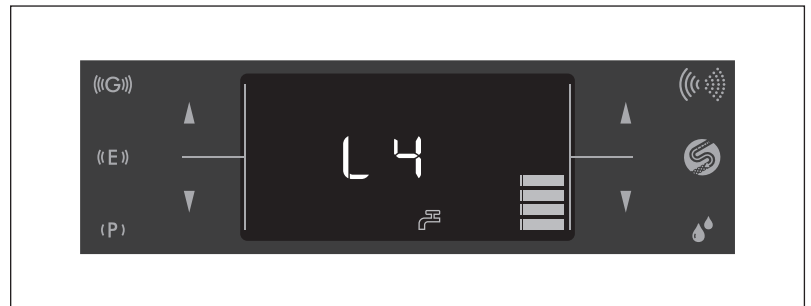- 1) Turn the POWER switch of the Control Unit to ON.
- 2) Touch the Operation Mode Select Key to select Powder Mode.
- 3) Touch and hold the Irrigation Mode Select Key for 2 seconds. Alarm sounds are heard and the LCD panel switches to the Basic Water Level adjustment mode.
- 4) Touch the Irrigation Volume Keys (▲/▼) to change the Basic Water Level ("L1" - "L5"). Adjust the Level as necessary.
- 5) Touch and hold the Irrigation Mode Select Key for 2 seconds to save the selected level to memory. An alarm sound is heard and the selected level is saved, and then the LCD Panel switches back to normal mode.

Basic Water Level adjustment mode [Water Level 4]

NOTICE • Basic Water Level returns to "L4" when this product is initialized to the factory defaults. Set the Basic Water Level again after initializing.

Reference

| Level | Water volume |
|--------------------|--|
| L5 |  Large |
| L4 (Initial value) | |
| L3 | |
| L2 | |
| L1 | |

12 Periodical Maintenance Checks

Refer to the check sheet below and perform periodical maintenance checks every three months. Contact your Authorized NSK Dealer if any abnormalities are found.

| Points to Check | Details |
|---|--|
| Ultrasonic Handpiece | Operate the Ultrasonic Handpiece to confirm there is no abnormal vibration, noise, heat, etc. Also check that the LED lights and water flows. |
| Chamber Cap | Check that there are no large/deep scratches. |
| Powder Spray | After filling the powder chamber, spray powder into a spittoon, etc., and confirm that powder is not leaking from connections other than the Nozzle Tip connection. |
| Water Spray | After filling the powder chamber, spray powder into a spittoon, etc., and confirm that water is not leaking from connections other than the Nozzle Tip connection. |
| Chamber | Check that there are no large/deep scratches. |
| Attaching/removing the Handpiece, Handpiece Hose, Irrigation Bottle and Chamber | Check that they can be attached and removed smoothly. If they are too loose or too tight, the O-rings may be damaged or deteriorated. Replace O-rings. (Refer to "10 Care and Maintenance.") |

Additional safety information:

Periodical safety testing shall be performed every two years following IEC 62353 (EN 62353), in accordance with test procedure OM-DE0947EN. As periodical safety testing is limited exclusively to professional dental installers who have been trained and qualified under the control of NSK, contact your authorized NSK dealer.

13 Error Code

"Error" and an Error Code are displayed on the LCD Panel of the Control Unit if this product stops operating due to malfunction, excessive pressure, disconnection, or incorrect operation. Depress the Foot Control button D to perform the error check again. If the problem is not detected, the error status is released and this product become operable. If the Error Code is still displayed, refer to the table below and take the necessary steps to resolve the problem.

| Error Code | Description of Error | Check/Remedy |
|------------|--|---|
| 0 | System Error | Turn on the Control Unit power again. Contact your Authorized NSK Dealer if the error does not disappear. |
| 1 | 24V voltage abnormality | |
| 2 | 12V voltage abnormality | |
| 3 | Pump current abnormality | Check the connection of the Irrigation Pump. If the problem still persists, contact your Authorized NSK Dealer. |
| 4 | Air pressure abnormality | Check the incoming air pressure. If the problem still persists, contact your Authorized NSK Dealer. |
| 5 | Heater voltage abnormality | Turn on the Control Unit power again. Contact your Authorized NSK Dealer if the error does not disappear. |
| 6 | Heater current abnormality | |
| 7 | Heater temperature abnormality | Check the water circuit and supply water to the Control Unit. Leave the Control Unit until it cools down before using it. Contact dealer if the error does not disappear. |
| 8 | Foot Control communication abnormality | Check the connection of the Foot Control. If the problem still persists, contact your Authorized NSK Dealer. |
| 9 | Ultrasonic Handpiece abnormality | Check the connection of the Handpiece. If the problem still persists, contact your Authorized NSK Dealer. |
| 12 | Touch sensor communication abnormality | Turn the POWER switch to OFF and then to ON. If the problem still persists, contact your Authorized NSK Dealer. |
| 13 | Error in self-check of the Ultrasonic Unit | |
| 14 | Ultrasonic Unit system error | |

14 Troubleshooting

If you are experiencing problems, please check the following before contacting your Authorized NSK Dealer.

(Ultrasonic Mode and Powder Mode)

| Problem | Cause | Remedy |
|---|---|---|
| Beeping sound | Power was turned on while the Foot Control was depressed. | Release the Foot Control and turn the POWER switch to ON. |
| | Abnormal heating of the Control Unit. | Stop operation and leave until Control Unit cools. |
| Touch panel does not react well or touched key is not recognized correctly. | Calibration is slightly misaligned due to static electricity, exogenous noise, etc. | Turn the POWER switch to OFF and then to ON. |
| | Key was not touched at the center. | Touch the center of the key. |
| <ul style="list-style-type: none"> Water does not stop The water level cannot be adjusted | Damage or wear of the O-Ring. | Follow the procedure in "10-1-5 Chamber Base Connector" to replace the O-Ring. |
| | Foreign material has become clogged in the water route. | Perform Auto-Cleaning. If the symptom is still not resolved, contact your Authorized NSK Dealer. |

(Ultrasonic Mode)

| Problem | Cause | Remedy |
|---|---|--|
| Alarm sounds are heard, and the Ultrasonic Mode Display (P/E/G) and "ULTRASONIC" blink alternately when the Foot Control button D is depressed. | The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece. | Wipe the sensor clean. |
| No/Poor vibration. | The AC power cord plug is not connected to the wall socket or electricity is not supplied to the wall socket. | Check the connection. |
| | The Fuse is burned out. | Refer to "10-5 Replacing the fuse" and replace the fuse. |
| | The Tip is not tightened firmly. | Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out. |
| | Worn Tip. | Replace the Tip. |
| | The screw parts of the Handpiece and Tip are not clean. | Wipe them clean. |
| | The screw parts of the Handpiece are worn out as a result of a long time of use. | Contact your Authorized NSK Dealer.* |
| | Power has not been correctly adjusted for the Tip. | Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or on the Tip case label. |
| | The Foot Control is disconnected. | Connect the Foot Control Cord Plug correctly. |
| | Failure of vibration device in the Handpiece. | Contact your Authorized NSK Dealer.* |
| | Failure of internal components of the Foot Control. | Contact your Authorized NSK Dealer.* |
| The Tip is bent or broken. | The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece. | Wipe the sensor clean. |
| | Power has not been properly adjusted for the Tip. | Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label. |
| The Tip falls off. | The Tip is not tightened firmly. | Tighten the Tip until the Tip Wrench clicks. |

| Problem | Cause | Remedy |
|--|--|--|
| Noise from the Handpiece. | Power has not been properly adjusted for the Tip. | Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label. |
| | The Tip is not tightened firmly. | Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out. |
| | Failure of vibration in the Handpiece or the Control Unit. | Contact your Authorized NSK Dealer.* |
| The Handpiece is overheating. | Power has not been properly adjusted for the Tip. | Adjust the power so it does not exceed the maximum power level mentioned in Tip-Power Guide or the Tip case label. |
| | The Tip is not tightened firmly. | Rotate the Tip Wrench until it makes a click sound and cannot rotate any further. If the problem still persists, replace the Tip Wrench as it may be worn out. |
| | The screw parts of the Handpiece and Tip are not clean. | Wipe them clean. |
| | The screw parts of the Handpiece are worn out as a result of a long time of use. | Contact your Authorized NSK Dealer.* |
| | The Irrigation Volume is insufficient or irrigation has stopped. | Irrigate at an adequate volume. |
| | Vibration failure in the Handpiece or the Control Unit. | Contact your Authorized NSK Dealer.* |
| No Irrigation supply and/or unstable Irrigation supply (when using the Bottle) | The Irrigation Tube is twisted. | Straighten the twisted Irrigation Tube. |
| | Time to replace the Irrigation Pump. | Replace with a new Irrigation Pump (Refer to 10-2 Changing the Irrigation Pump). |
| No/Poor water supply. (when using Tap Water) | Water is not supplied to the Control Unit. | Check the water circuit so that water is supplied to the Control Unit. Supply water pressure: 0.1 - 0.3MPa |
| | The Irrigation Volume is set to "0." | Adjust to the adequate Irrigation Volume using the Irrigation Volume setting. |
| | Water supply may be unstable when the Irrigation Volume is insufficient. | This is not a malfunction. Increase the Irrigation Volume in the Irrigation Volume setting. |
| | Water Filter is clogged with foreign substances. | Replace the water filter with a new one. |
| Water leakage. | The Irrigation Tube is not connected correctly. | Firmly insert the Irrigation Tube all the way into the Irrigation Connector. |
| | O-Ring at the Handpiece Cord is worn or damaged. | Replace with a new O-ring by following the procedures in "10-1-1 Handpiece Cord." |
| | The water circuitry in the Control Unit is damaged. | Contact your Authorized NSK Dealer.* |
| Handpiece LED does not light. | The Handpiece is not connected to the Handpiece Cord correctly. | Firmly insert the Handpiece all the way into the Handpiece Cord. |
| | Failure in the Control Unit. | Contact your Authorized NSK Dealer.* |

(Powder Mode)

| Problem | Cause | Remedy |
|---|--|--|
| Alarms are heard, and "POWDER" and the tooth icon blink alternately when the Foot Control button D or B is depressed. | The sensor built in to the Handpiece Holder is dirty and mistakenly detects mounting of the Handpiece. | Wipe the sensor clean. |
| | Chamber is not attached securely. | Attach the Chamber securely until the confirmation sound is heard. |
| Confirmation sound cannot be heard, or the icon that indicates Prophy Mode or Perio Mode is not displayed when the Chamber is attached. | Failure in the Control Unit (disconnection). | Contact your Authorized NSK Dealer.* |

| Problem | Cause | Remedy |
|--|---|---|
| Air and powder are not supplied. Or air and powder are supplied but the pressure is not strong enough and cleaning power is also weak. | The inside of the Nozzle Tip is clogged with powder. | Replace the Nozzle Tip. |
| | The inside of the Handpiece is clogged with powder. | Clean the Nozzle base with the supplied Cleaning wire (small) and Nozzle end with the cleaning file. In addition, clean the inside of the Handpiece with the Cleaning wire (large). If powder and air do not flow after being cleaned as described above, perform Ultrasonic Cleaning with warm water. (Refer to "9-3 Ultrasonic cleaning (weekly).") |
| | The inside of the Chamber is clogged with powder. | Remove powder from inside the Chamber by following the procedures in "9-1-5 Chamber and Handpiece Hose". |
| | Powder has partially solidified with water, etc., in the air passage. | Remove powder from inside the passage by following the procedures in "9-1-4-2 Auto-Cleaning." |
| | Powder container has been left open exposing powder to the air. | Dispose of the powder and use new powder. |
| | Water is mixed in supply air. | Open the filter in the air supply circuit to drain the compressor, and check that water has not accumulated inside the air filter and compressor drain. |
| | The VCP Element is clogged. | Contact your Authorized NSK Dealer.* |
| Water leaks from between the Handpiece and Handpiece Hose. | The O-ring is scratched. | Replace the O-ring in the Handpiece joint section, apply the supplied turbine oil and rub it in with your finger. (Refer to "10-1-2 Handpiece Hose (for Prophy use and Perio use).") |
| Cannot remove the Chamber because the Lock Button cannot be pressed | The button is locked | Follow the procedure in "Releasing the Lock Button" in "9-1-5 Chamber and Handpiece Hose" to release the Lock. If the symptom is not resolved, contact your Authorized NSK Dealer.* |
| Cannot close the Chamber Cap. | Powder is adhered to the threads of the Chamber and/or the Chamber Cap. | Clean the threads of the Chamber Cap (Refer to "Cleaning Chamber Cap" in "9-1-5-2 Cleaning and disinfection"). |

* Cannot be fixed at clinics, etc.

15 Specifications

<Control Unit>

| | |
|-------------------------------------|---|
| Model | Varios Combi Pro |
| Power source | AC100V - 240V 50/60Hz |
| Drive frequency | 28 - 32kHz |
| Power input | 78 - 98VA |
| Maximum output | 11W |
| Supply air pressure | 0.55 - 0.75MPa (5.5 - 7.5kgf/cm ²) |
| Supply water pressure | 0.1 - 0.3MPa (1.0 - 3.0kgf/cm ²) |
| Bottle volume | 400ml |
| Irrigation Volume (Ultrasonic Mode) | 5.0 - 70.0mL/min (Irrigation Mode: Bottle/Tap water, Supply water pressure: 0.2MPa) |
| Dimension | W225 × D290 × H162mm (Including the Irrigation Bottle and Chamber) |
| Weight | 2.7kg (Irrigation Bottle and Chamber not included) |

<Foot Control>

| | |
|-------------|---|
| Model | FC-79 |
| Cord length | 2.5m |
| Dimensions | W204 × D157 × H128mm (including Hanger) |
| Weight | 415g (including Hanger) |

<Ultrasonic Handpiece>

| | |
|---------------------|---------------------------------|
| Model | VA2-LUX-HP |
| Cord length | 1.8m (Connector not included) |
| Type of oscillation | Piezo type |
| Light | White LED |
| Dimensions | φ20 × 103mm (cord not included) |
| Weight | 54g (cord not included) |

<Powder Handpiece>

| | |
|-------------|---------------------------------|
| Model | PROPHY-HP/PERIO-HP |
| Hose length | 1.8m (Connector not included) |
| Dimensions | φ16 × 107mm (cord not included) |
| Weight | 33g (cord not included) |

| | Temperature | Humidity | Atmospheric pressure |
|--|-------------|------------|----------------------|
| Use environment*2 | 0 - 40°C | 35 - 75%*1 | – |
| Transportation and Storage environment | –10 - 50°C | 10 - 85% | 500 - 1,060hPa |

*1 Without condensation

*2 The use of this product under conditions other than specified above may cause malfunction.

16 Classification of Equipment

- Type of protection against electric shock:
 - Class I equipment
- Degree of protection against electric shock:
 - Type B applied part:  (Applied parts: Ultrasonic Tip, Ultrasonic Handpiece, Nozzle Tip, Powder Handpiece)
- Method of sterilization or disinfection recommended by the manufacturer:
 - Refer to 9. Post-use Maintenance
- Degree of protection against ingress of water as detailed in the current edition of IEC 60529:
 - Foot Control: IPX1 (Protected against vertically falling water drops)
- Degree of safety of application in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide:
 - Equipment NOT suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Mode of operation:
 - Continuous operation

17 Operation Principle

Ultrasonic Mode

A sinusoidal electrical signal is delivered by the generator at ultrasonic frequency ($f > 20\text{kHz}$). This signal is applied to the 'piezoelectric ceramic' located inside the transducer. Piezoelectric ceramic converts this signal into mechanical vibrations. These vibrations are at the same ultrasonic frequency as the electrical signal. The mechanical vibrations are propagated towards the distal end of the transducer. The "TIP" insert, which is attached at the distal end of the transducer, vibrates at ultrasonic frequencies and to achieve the purpose of the equipment.

Powder Mode

Sprays powder using air pressure to clean the tooth surface.

18 Symbol

| | | | |
|---|--|---|---|
|  | This product can be sterilized in a steam sterilizer at 135°C. |  | Batch Code |
|  | Conforms to CE European Directive of "Medical device directive 93/42/EEC." |  | Serial number |
|  | Authorized representative in the European community. |  | Caution |
|  | Consult operation instructions. |  | Temperature |
|  | Protected against dripping |  | Drinkable tap water |
|  | Marking on the outside of Equipment or Equipment parts that include RF transmitters or that apply RF electromagnetic energy for diagnosis or treatment |  | Lamp / lighting / illumination |
|  | Do not reuse. |  | Visual check |
|  | Keep Dry. |  | Ultrasonic bath |
|  | EOG sterilization |  | Humidity limitation |
|  | GS1 DataMatrix for Unique Device Identifier. |  | Atmospheric pressure limitation |
|  | This product can be washed using a washer-disinfector. |  | Catalog number (Order code) |
|  | Manufacturer. |  | Medical Device |
|  | Follow the waste of electric and electronic equipment (WEEE) Directive (2012/19/EU) for product and accessory disposal |  | Unique Device Identifier |
|  | Type B applied part. |  | Authorized representative for Switzerland |
|  | For indoor use only. |  | Date of manufacture |
|  | Temperature limitation |  | Importer |
|  | Use-by date | | |

19 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

20 Option Parts List

| Model | REF | Remarks |
|----------------------------------|------------|--|
| MG-4H Multi Gauge | Z109400 | |
| Tip Holder | Z221080 | |
| Perio Set | Y1003042 | Powder Handpiece (For Perio use) Chamber (For Perio use) Handpiece Hose (For Perio use) Perio-Mate powder Perio-Mate Nozzle Tip (40 pieces) Perio-Mate Nozzle Tip Remover |
| Powder Handpiece (For Perio use) | T1077001 | |
| Chamber (For Perio use) | U1135019 | |
| Handpiece Hose (For Perio use) | T1053001 | |
| Nozzle (For Perio use) | T1077012 | |
| Perio-Mate powder | Y900938 | 2 bottles, each bottle contains 160g |
| Perio-Mate Nozzle Tip | Y1002741 | 40 pieces |
| Perio-Mate Nozzle Tip Remover | T1043751 | |
| Water Pressure Reducing Valve | U1135093 | |
| CLA-20 | ZA11900108 | Cleaning Adaptor for Perio Handpiece / Prophy Handpiece. |

21 Spare Parts List

| Model | REF | Remarks |
|-------------------------------------|----------|---|
| FC-79 | Z1256001 | Foot Control |
| VCP Bottle Set | U1135017 | Bottle and Cap (Irrigation Bottle, Irrigation Bottle Cap) |
| VCP Bottle Cap | U1135116 | Cap only (Irrigation Bottle Cap) |
| VCP Bottle (Bottle only) | U1135099 | Bottle only (Irrigation Bottle) |
| O-ring set (Irrigation Bottle) | Y1003544 | One each of Thick and Thin O-ring |
| O-ring | Y1003546 | 1 for the Handpiece Cord |
| O-ring set (Chamber Base Connector) | Y1003435 | |
| O-ring set (Handpiece Hose) | Y1003500 | One each of Thick and Thin O-ring Common Parts for Prophy use and Perio use |
| O-ring set (Nozzle) | Y1003501 | One each of Thick and Thin O-ring Common Parts for Prophy use and Perio use |
| O-ring | Y1003545 | For Chamber Cap |
| Handpiece Cord | E351054 | |
| Ultrasonic Handpiece | E351050 | |
| Sterilization case | Z1035001 | For Ultrasonic parts |
| Tip Wrench (CR-10) | Z221076 | |
| Water Filter Set | U1135078 | |
| Water Filter | U387042 | |
| Spanner Wrench (5x8) | Y1001301 | 2 |
| Irrigation Pump | U1135079 | Connector ring excluded |
| FLASH pearl | Y900698 | 4 bottles, each bottle contains 300g |
| Prophy Set | Y1003771 | Powder Handpiece (For Prophy use) Chamber (For Prophy use) Handpiece Hose (For Prophy use) FLASH pearl (5 sachets) Replacement O-ring |
| Chamber (For Prophy use) | U1135018 | |
| Handpiece Hose (For Prophy use) | T1051001 | |
| Powder Handpiece (For Prophy use) | T1074001 | |
| Nozzle (For Prophy use) | T1074012 | |
| Chamber Cap | U1135089 | Cap only |
| Cleaning File | T798092 | For the distal end and inside the Nozzle |
| Cleaning Wire (S) | T798095 | For the root of the Nozzle |
| Cleaning Wire (L) | T798091 | For parts other than those above |
| Cleaning Brush | T798093 | |

| Model | REF | Remarks |
|----------------|-------------|----------------------|
| Unlocking tool | U1135256 | |
| Ring Wrench | T1074601 | |
| Turbine Oil | Z016112 | |
| Air Hose | D0914406036 | |
| Fuse | D1202220010 | Rated value T2AH250V |

22 Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

23 EMC Information (Electromagnetic Compatibility Information)

| Guidance and manufacturer's declaration - Electromagnetic Emissions | | |
|---|------------------------|--|
| The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment. | | |
| Emissions test | Compliance | Electromagnetic environment - guidance |
| RF emissions CISPR11/EN55011 | Group 1 Class B | The product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| RF emissions CISPR11/EN55011 | Group 1 Class B | |
| Harmonic emissions EN/IEC61000-3-2 | class A (except 120V) | |
| Voltage fluctuations/flicker emissions EN/IEC61000-3-3 | Complies (except 120V) | |

| Guidance and manufacturer's declaration - Electromagnetic Immunity | | | |
|---|---|---|--|
| The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment. | | | |
| Immunity test | EN/IEC60601 test level | Compliance level | Electromagnetic environment - guidance |
| Electrostatic discharge (ESD) EN/IEC61000-4-2 | ±(2,4)6kV contact ±(2,4)8kV air | ±(2,4)6kV contact ±(2,4)8kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. |
| Electrical fast transient/burst EN/IEC61000-4-4 | ±2kV for power supply lines ±1kV for input/output lines | ±2kV for power supply lines ±1kV for input/output lines | Mains power quality should be that of a typical commercial or hospital environment. |
| Surge EN/IEC61000-4-5 | ±1kV line(s) to line(s) ±2kV line(s) to earth | ±1kV line(s) to line(s) ±2kV line(s) to earth | Mains power quality should be that of a typical commercial or hospital environment. |
| Voltage dips, short interruptions and voltage variations on power supply input lines EN/IEC61000-4-11 | <5% Ut (>95% dip in Ut) for 0.5 cycles 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 0.5 cycles <5% Ut (>95% dip in Ut) for 5 sec | <5% Ut (>95% dip in Ut) for 0.5 cycles 40% Ut (60% dip in Ut) for 5 cycles 70% Ut (30% dip in Ut) for 0.5 cycles <5% Ut (>95% dip in Ut) for 5 sec | Mains power quality should be that of a typical commercial or hospital environment. If the user of the product requires continued operation during power mains interruptions, it is recommended that the product be powered from an uninterruptible power supply or a battery. |
| Power frequency (50/60Hz) magnetic field EN/IEC61000-4-8 | 3A/m | 3A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| NOTE: 'Ut' is the AC mains voltage prior to application of the test level. | | | |

| Guidance and manufacturer's declaration - Electromagnetic Immunity | | | |
|--|--------------------------|------------------|---|
| The product is intended for use in the electromagnetic environment specified below. The customer or the user of the product should assure that it is used in such an environment. | | | |
| Immunity test | EN/IEC60601 test level | Compliance level | Electromagnetic environment - guidance |
| Conducted RF EN/IEC61000-4-6 | 3Vrms 150kHz to 80MHz | 3Vrms | Portable and mobile RF communications equipment should be used no closer to any part of the product, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz to 800MHz $d = 2.3 \sqrt{P}$ 800MHz to 2.5GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, and (d) is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters as determined by an electromagnetic site survey(a) should be less than the compliance level in each frequency range(b). Interference may occur in the vicinity of equipment marked with the following symbol:  |
| Radiated RF EN/IEC61000-4-3 | 3V/m 80MHz to 2.5GHz | 3V/m | |
| NOTE 1: At 80MHz and 800MHz, the higher frequency range applies. | | | |
| NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. | | | |
| a: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the product is used exceeds the applicable RF compliance level stated above, the product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the product. | | | |
| b: Over the frequency range 150kHz to 80MHz, the field strength should be less than 3V/m. | | | |

| Cables and accessories | Maximum length | Complies with |
|------------------------|-------------------|---|
| Handpiece Cord | 2.0m (Unshielded) | RF emissions, CISPR11 ClassB/Group1 |
| Foot Control Cord | 2.5m (Unshielded) | Harmonic emissions, (except 120V) EN/IEC61000-3-2 |
| AC Power Cord | 3.0m (Unshielded) | Voltage fluctuations/flicker emission, (except 120V) EN/IEC61000-3-3 Electrostatic discharge (ESD): EN/IEC61000-4-2 Electrical fast transient/burst: EN/IEC61000-4-4 Surge: EN/IEC61000-4-5 Voltage dips, short interruptions and voltage variations on power supply input lines: EN/IEC61000-4-11 Power frequency(50/60Hz) magnetic field: EN/IEC61000-4-8 Conducted RF: EN/IEC61000-4-6 Radiated RF: EN/IEC61000-4-3 |

Recommended separation distances between portable and mobile RF communications equipment and the product

The product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the product as recommended below, according to the maximum output power of the communications equipment.

| Rated maximum output power of transmitter | Separation distance according to frequency of transmitter m | | |
|---|---|---------------------------------------|--|
| | 150kHz to 80MHz $d = 1.2 \sqrt{P}$ | 80MHz to 800MHz $d = 1.2 \sqrt{P}$ | 800MHz to 2.5GHz $d = 2.3 \sqrt{P}$ |
| 0.01 | 0.12 | 0.12 | 0.23 |
| 0.1 | 0.38 | 0.38 | 0.73 |
| 1 | 1.2 | 1.2 | 2.3 |
| 10 | 3.8 | 3.8 | 7.3 |
| 100 | 12 | 12 | 23 |

For transmitters rated at a maximum output power not listed above, the recommended separation distance “d” in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where “P” is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80MHz and 800MHz, the higher frequency range applies.

NOTE2 : These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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NSK Europe GmbH